Overbed Table

Model No: 770-872

ASSEMBLY INSTRUCTIONS

Open the end of the box and slide out all components.
Lay the components on a level surface and remove the base, table top and hardware.

Install the casters (7 & 8) as shown.

ENSURE CHROME SCREW (13) IS FULLY FASTENED PRIOR TO INSTALLATION.


Insert column assembly (2) into the bracket (4) to attach to the table top, as shown. Be sure the trip rod (inside the column) is fully inserted into the column, align the holes of the column assembly (2) and bracket (4) and install and fasten the two screws (5).

Place 2 bolts (11) through the bolt plate (10) and bolt the sleeves (9) then place the bolt assembly through the base (6) into to the column assembly (2). Tighten bolts.

CAUTION: For safety reasons do not remove the chrome hex-head screw (13) on the column, just loosen slightly after assembly.

OPERATING INSTRUCTIONS:

- To raise the Table top:** apply light pressure upward from under the table top.
- To lower the Table top:** squeeze the trip lever (3) upwards towards the table top and push down on the table top at the same time, until it reaches the desired height. Release the trip lever (3). **Note: When lowering the table top put pressure on the end of the table top closest to the column and not the end furthest from the column.**
- Once the table is assembled, prior to use, remove the protective film from the Table top (1).

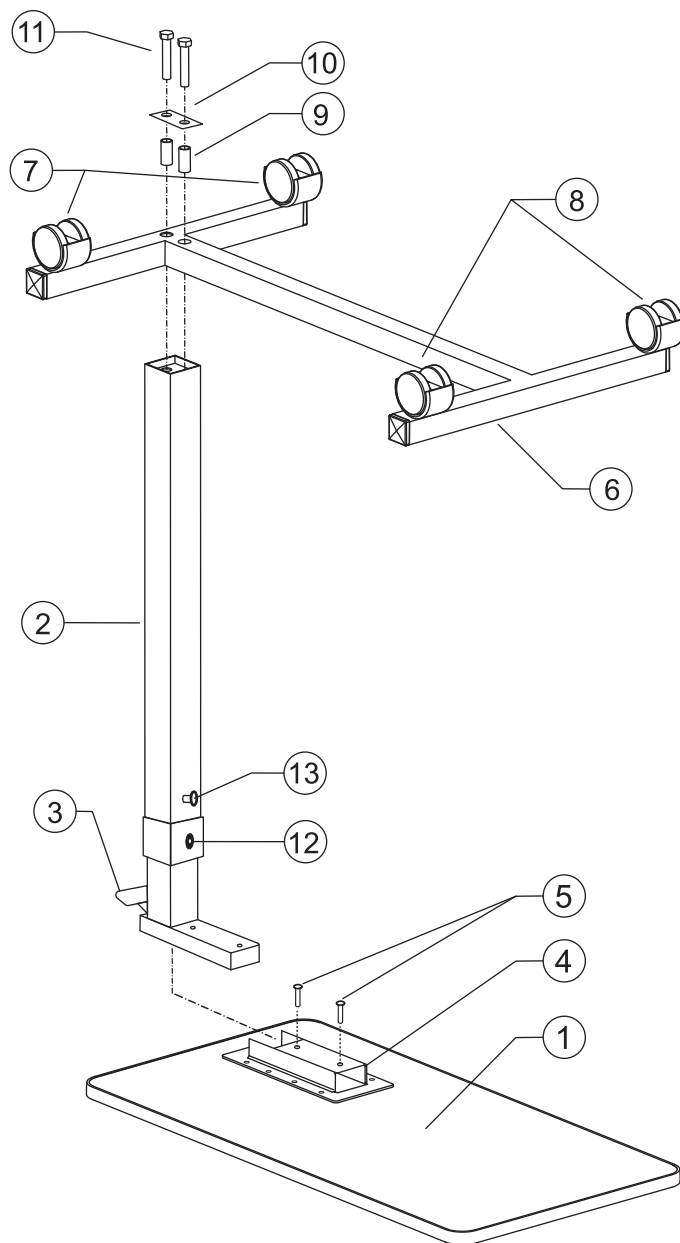PARTS LIST

1 - Table Top	6 - Base	11 - 3/8-15 inch Bolt (2)
2 - Column Assembly	7 - Casters with Stopper (2)	12 - Black Screw
3 - Trip Lever	8 - Casters without Stopper (2)	13 - Chrome Screw (Hex-Head Screw)
4 - Bracket to attach table top to column	9 - Bolt Sleeves (2)	14 - Tool kit: 4mm Allen key (1),
5 - Screws (2)	10 - Bolt Plate	5mm Allen key (1), Closed wrench (1)

To learn more about this or any other **A.M.G.** product, please contact your **A.M.G. Medical Inc.** representative or call toll-free: 1-800-363-2381.

A.M.G. Medical Inc.
Montréal, Québec
H4T 1V5 Canada
www.amgmedical.com

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