For other details of Parsons A.D.L. Inc. products for the elderly and disabled contact your dealer or visit our website:

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www.parsonsadl.com

Limited One Year Warranty

Parsons A.D.L. Inc. warrants to the original owner of this product that it is free from defects in material and workmanship for one year from date of purchase. We will repair or replace any part of the product, which in our opinion is defective, provided the product has not been abused, misused, altered or damaged after purchase. (This includes damage due to use of tools or harsh chemicals.)

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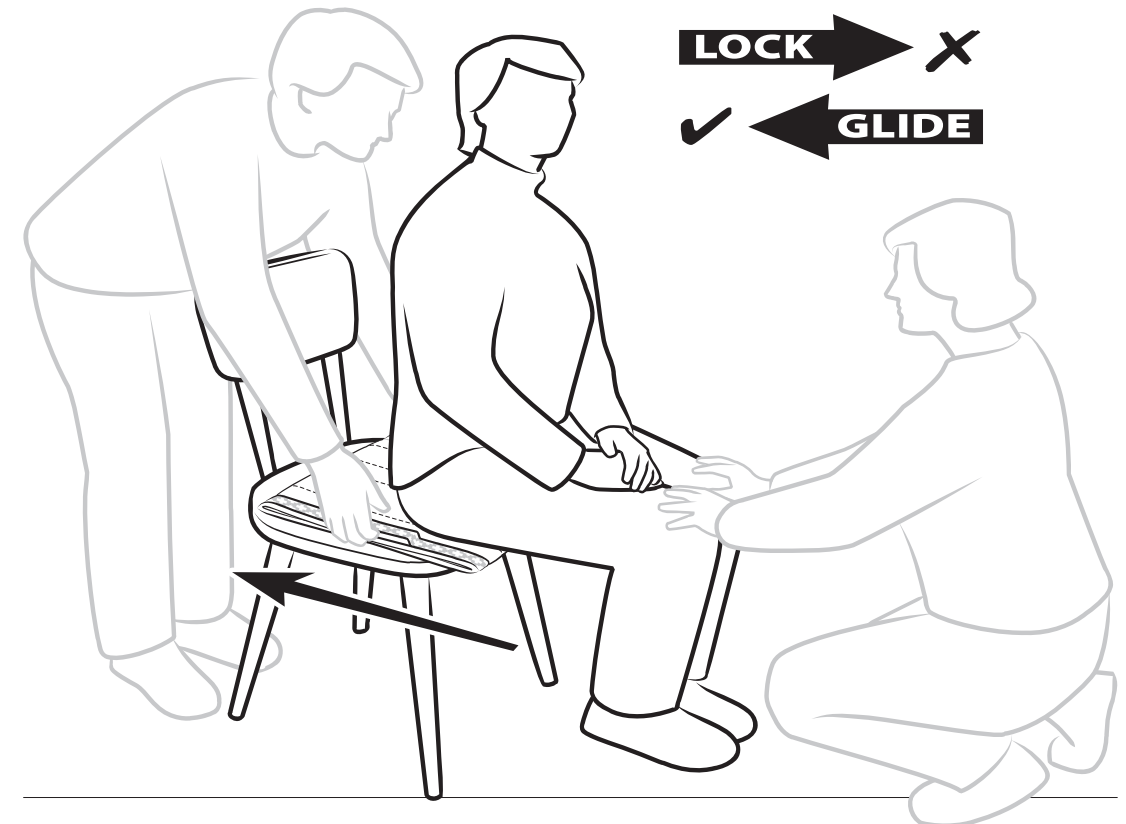
16" x 20" (40 x 50cm)

Standard Glide Lock Sheet for standard chair or wheelchair

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PK-125

Glide Lock Sheet

A Glide Lock Sheet is a positioning device that slides in one direction but locks in the other when the user is sitting on it. Glide Lock Sheets are cushioned for added comfort. The top surface is a breathable, waterproof material that allows someone to sit on it for extended periods of time without building up body moisture. The bottom surface is an anti-slip material to securely grip the chair's upholstery.

For best results:

Insert the Glide Lock Sheet prior to seating the user. If necessary, the caregiver can roll the user to one side then insert the device under the user.

Make sure the directional arrows on the label are pointing in the correct direction.

The anti-slip material on the bottom of the device goes to the back of the chair to prevent the device from sliding away when the caregiver adjusts the user's position.

Loop handles on the side are to assist the user or the caregiver with positioning. Positioning can be done from the front or back as illustrated on the front page.

The user should be sitting as central as possible on the Glide Lock Sheet. If needed, reposition the user and make sure they sit in the middle of the Glide Lock Sheet. Whenever possible ask the user to help the caregiver with this.

Once seated on the Glide Lock Sheet the user can be moved towards the back of the chair and will not slide forward.

To remove the Glide Lock Sheet either maneuver the user to the side of the chair or let him stand up, or log roll the user off the device.

Washing instructions: Wash in lukewarm water, tumble dry at low temperatures or air dry. No bleach or ironing as this will damage the materials that make the Glide Lock functions work.

NOTE: Please use in accordance with your organization's policies and procedures. Do not leave the device near exposed or naked flames. Always ensure that this device is the appropriate equipment for the task of moving a particular user.

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