
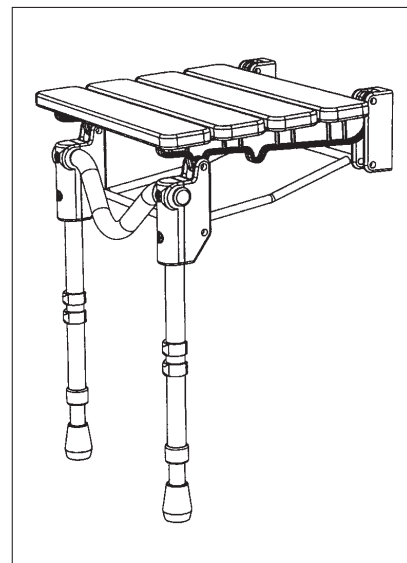
NEW CODE	OLD CODE	US CODE
09 107 6801	AA1690	08 130 5382
09 107 6850	AA1694	08 130 5390
09 107 6884	AA1697	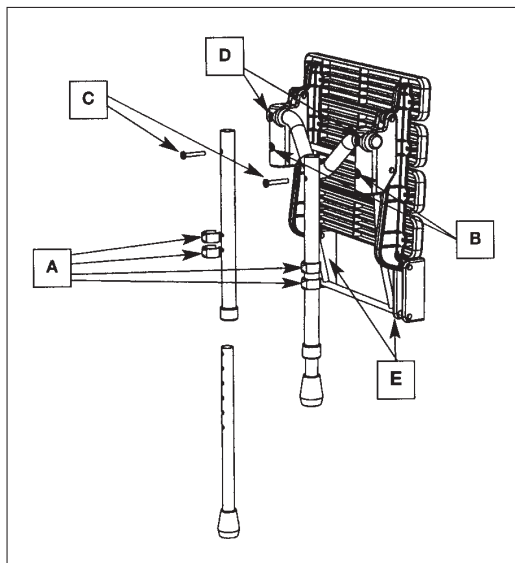

Intended uses

- To compensate for a disability by providing a seat for a person taking a shower.

Instructions for use

- The seat can be folded up out of the way when not in use. Be sure the latch holds the seat securely up.
- To lower the seat, hold onto the cross bar between screws D, and pull out from the wall.

Cautions

- DO NOT PULL ON THE LEGS TO LOWER OR RAISE THE SEAT.**
- Shower seats should be installed by a qualified professional.
- Shower seats should not be attached to plastic shower walls, stud partitioning, plasterboard-type cavity walls, or any similar type of shower surround or wall.
- The shower seats are not to be used to support more than 158kg / 348lb. / 24 $\frac{3}{4}$ st.
- The seats are not to be used with water that is colder than 5°C / 40°F or hotter than 50°C / 120°F.
- Be sure the seat is securely in place before putting weight on it.
- Use of a slip-resistant bath mat is strongly recommended.
- Cover the seat with a towel to help reduce slipperiness.

Instructions for installation

- These seats should be installed by a qualified professional, who will choose hardware appropriate for the specific installation. **Note that each individual fixing should resist a shear force of 60 kg / 130 lb. and a pull-out force of 20 kg / 45 lb.**
- Remove the screws that are labelled C in the drawing from each block (B) and loosen the screws that are labelled D.
- Slide the legs into the blocks and tighten screws D. Screws C can be left out at this stage.
- Establish the height of the seat, which can be adjusted from 460 to 590 mm / 18 to 23 inches as follows:
 - With the seat in the horizontal position, position the rear brackets (E) against the shower wall.
 - To adjust leg length, remove the four pin clips (A) from the legs, determine the appropriate length, and replace the pin clips.
 - Be sure both legs are adjusted to the same height. A gap up to 5 mm / $\frac{3}{16}$ " under one leg is acceptable. However, if the angle of the shower tray results in a greater gap, the seat must be installed at an angle to compensate for it.

- Mark the lower securing holes through the rear brackets. All fixing holes in the rear bracket must be used. Remove the seat from the wall and drill and plug the wall.

- Loosen the screws D and remove the legs from the seat.
- With the seat locked in the vertical position, position the rear brackets over the holes and fasten the seat to the wall.
- Slide the legs back into the blocks and line up the leg securing holes. Replace and tighten screws C, and tighten screws D in the blocks.

Care and Cleaning

The seat can be cleaned with mild detergent and water.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Performance Health International Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

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