





# Meywalk®4 the dynamic walker

The users of the new Meywalk<sup>®</sup> 4 are typically wheelchair dependent persons of all ages that require a high degree of aid and support to stand up and walk.

At the same time Meywalk<sup>®</sup> 4 offers the user much more quality of life in the shape of enabling the user to take part in everyday activities and make decisions by him- or herself.

Meywalk<sup>®</sup> 4 is the fourth generation of the well-known Meywalk<sup>®</sup> product family that



is now presented with new and improved features to further aid and support the user in the daily life.

#### The springy walker

Meywalk<sup>®</sup> 4 stands out in the market of assistive technology by being spring suspended. The springs stimulate the muscles of the user and the movements that are a part of a natural gait are imitated. This way the gait pattern is developed. Furthermore it has been observed that the suspension system very often motivates users with mental disabilities to move around and thus create a platform for further development.

In practice the upper part of the frame follows the movements of the user and provides optimal support during the various phases of gait.

#### Dynamic saddle seat

A new feature on Meywalk<sup>®</sup>4 is the dynamic saddle seat that also follows the movements of the user. The rubber suspended seat allows for a certain degree of hip rotation during gait. When the user walks the saddle seat follows the sideways movements of the hips, but at the same time the user is always brought back to the starting position during the various phases of gait in a gentle and safe manner. The dynamic seat is unique to the Meywalk<sup>®</sup> 4 and is not known from earlier generations of the Meywalk<sup>®</sup>-family.

#### Fighting gravity

The dynamic seat and the padded trunk support of the walker support the user and help to keep the body erect. Both seat and trunk support are continuously adjustable in height. This way the walker can be fitted precisely to the user and allows for growth.

Furthermore it is possible to adjust the angle of the seat and trunk support so that the user can place him- or herself anywhere between a vertical and slightly forward position if this proves to be necessary to support the gait function.

#### **Facilitated access**

As known from the previous Meywalk<sup>®</sup> models entering and exiting Meywalk<sup>®</sup> 4 is quite easily done by way of the built-in swing bar that lowers and raises the seat and trunk support when the assisted user enters or exits the walker.

#### **Tool-free adjustments**

Meywalk<sup>®</sup> 4 is available in two sizes and can be adjusted to fit the needs and size of each individual user. All settings are done easily without the use of tools, which is a great advantage during active and busy days. Most surfaces on the walker are either coated or upholstered with closed-surface polyurethane that ensure longevity, easy maintenance and cleaning.





#### **Heavy-duty**

Many have wished for a walker of this type for those users that weigh more than 100 kg. This has now been made possible with the Meywalk<sup>®</sup> 4 as it has been tested and approved for users with a body weight of up to 130 kg.

Meywalk<sup>®</sup> 4 comes standard with dynamic saddle seat and rear stop, handle bar, trunk support in the required size, parking- and drag brakes, non-reverse function, front wheel swivel locks and springs that match the user's body weight.

The trunk support for Meywalk<sup>®</sup> 4 is available in five sizes: Circumference 70 cm, 90 cm, 105 cm, 120 cm and 130 cm.

The springs are available in five variations: White springs for a body weight of min. 20 kg, yellow for min. 45 kg, blue for min. 65 kg, red for min. 80 kg and silver for a body weight of min, 100 kg.

### Meywalk<sup>®</sup> 4 enables the user to take part in everyday activities



#### The right match

An extended range of accessories are available for Meywalk<sup>®</sup> 4. The accessories ensure even more that the walker will match the specific needs of the user:

- : : Trunk pad
- : Hip pads with trunk pad
- :: Leg divider
- :: Leg guides
- :: Small swing bar for Medium
- : : Anti-tip support











## meywalk<sup>®</sup> 4



If you wish to try the Meywalk® 4 please provide us or your local dealer with the following user information: inner leg length, body weight and chest circumference.







Inner leg length

Chest circumference

Medium	Large
93 cm	110 cm
62 cm	68 cm
h): 50 cm	57 cm
60 - 84 cm	80 - 104 cm
101 - 133 cm	120 - 152 cm
130 kg	130 kg
32 kg	34 kg
Metallic red	Metallic black
011200	011250
	93 cm 62 cm 50 cm 60 - 84 cm 101 - 133 cm 130 kg 32 kg Metallic red

\* Heights are measured with half compression of springs.

## CE

Meywalk® 4 is a patented design and is CE-marked and complies with the corresponding norms. Meywalk®4 is made of steel and the upholstery is made of polyurethane.



Meyland-Smith A/S · Industrivej 27 · DK-9830 Taars Tlf.: +45 98 96 19 85 · Fax.: +45 98 96 19 86 www.meyland-smith.dk