



EZ-ON PRODUCTS

1-800-323-6598

www.ezonpro.com

100WC

WHEEL CHAIR MOUNT

IMPORTANT INFO

- Wheelchair Mount is strictly a positioning device.
- Adaptable to most wheelchairs.
- Helps to ensure proper posture in wheelchair.
- Additional tie downs are necessary to secure wheelchair to vehicle.
- Additional passenger restraint is necessary to safely secure passenger in a moving vehicle.
- Use with EZ-ON Vest only.
- Register this product to be notified for a safety recall.

WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.

WARNING


Death or serious injury can occur If manufacturer's instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

EMERGENCY RELEASE PROCEDURE

When evacuating the passenger in a safety vest, the vest webbing is never cut.
Make one cut on both straps on the backside of the wheelchair.



**100 SCBC
EZ-ON Safe Cut
Belt Cutter**





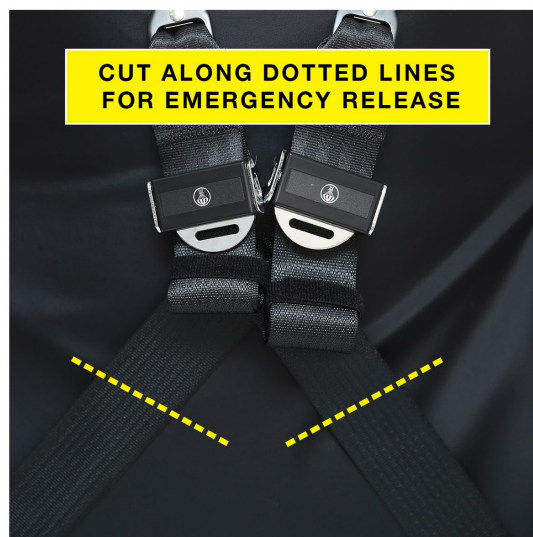




100WC

WHEEL CHAIR MOUNT

INSTALLATION INSTRUCTIONS

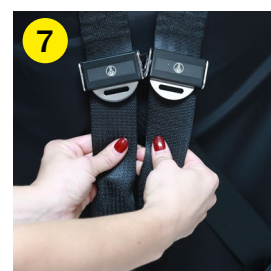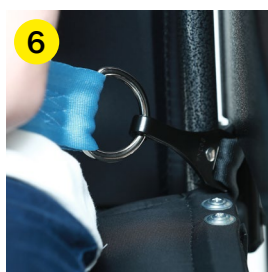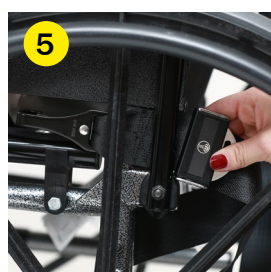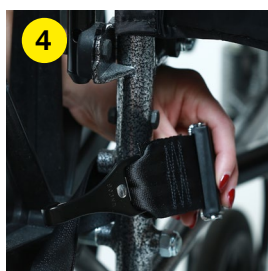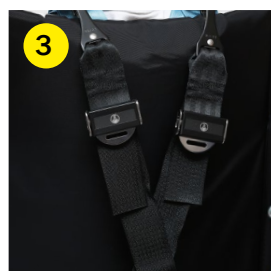

1. Sit passenger in wheelchair.

* Make sure hips are as close to the wheelchair back as possible.

2. Attach snap hooks to the shoulders of the vest.

3. Cross straps in the back of the wheelchair.

4. One side at a time wrap strap around wheel chair frame.

5. Slide hook into the front of seat.

6. Attach snap hooks to D-rings

7. Tighten each strap by pulling on excess webbing at adjuster.