

The pad slips over the top of a full side rail and goes across and under the mattress to help prevent bed rail entanglement. A decubitus fur liner protects the patient who bangs against the hard metal bars. Rails may be raised or lowered without removing the side rail pad, and reinforced holes allow wrist restraints or belts to be secured to the bed frame. Machine washable. Synthetic fur lined.

REF 5702 82"L x 65"W (208 cm x 165 cm)

Monitoring

⚠ WARNING Posey Full Side Rail Cover will not prevent a determined patient from exiting the bed, going over or attempting to go between or through side rail gaps. DO NOT articulate bed if product is applied. **ALWAYS** follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

Application Instructions:

1. Inspect product prior to use. DO NOT use soiled or damaged products.
2. Remove mattress from bed.
3. Slip side rail covers over both side rails, fur side towards the patient.
4. Replace mattress and secure hook and loop fasteners at the ends of the side rails and to the movable part of the bed frame.
5. Check for proper fit. Modified beds may not be compatible with the Posey Full Side Rail Cover.

Washing Instructions

160°F
71°C
WASH HOT
25 MIN.

CL
BLEACH AS DIRECTED
ON CONTAINER

☐
DRY ON
LOW



Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage And Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • www.posey.com
Phone: 1.800.447.6739 or 1.626.443.3143 • Fax: 1.800.767.3933 or 1.626.443.5014