# MAXISLIDE

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# Introduction

---

The vertical and horizontal lines printed in the margins adjacent to the text/illustrations in these instructions are for ArjoHuntleigh use only and should be disregarded by the reader.

Some of the information contained in these instructions may become outdated, due to improvements made to the product in the future. If you have any questions regarding these instructions or your MaxiSlide™, please contact ArjoHuntleigh or their approved distributor.

© ArjoHuntleigh 2012

*Our policy is one of continuous development and we therefore reserve the right to make technical alterations without notice. The content of this publication may not be copied either whole or in part without the consent of ArjoHuntleigh.*

*ArjoHuntleigh strongly advise and warn that only ArjoHuntleigh Designated Parts, which are designed for the purpose, should be used on equipment and other appliances supplied by ArjoHuntleigh, to avoid injuries attributable to the use of inadequate parts.*

*Unauthorised modification of any ArjoHuntleigh equipment may affect its safety and are in breach of any warranty on it. ArjoHuntleigh will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorised modification to its products.*

*ArjoHuntleigh's Conditions of Sale make specific provision confirming no liability in such circumstances.*

## Contents

---

| SECTION                                                                         | Page No. |
|---------------------------------------------------------------------------------|----------|
| Foreword.....                                                                   | 3        |
| Safety Instructions.....                                                        | 3        |
| Using your MaxiSlide .....                                                      | 4-7      |
| General .....                                                                   | 4        |
| Carl, Doris & Emma - Fitting 2 MaxiSlides using the unravelling technique. .... | 4        |
| Doris - Turning in bed with two people ....                                     | 5        |
| Doris - Moving up the bed, two people, pulling .....                            | 6        |
| Emma - Lateral transfer onto trolley .....                                      | 6        |
| Moving out of a confined space on the floor...                                  | 7        |
| Care of your MaxiSlide .....                                                    | 8        |
| Ordering Details .....                                                          | 9        |

# Foreword

---

Thank you for purchasing ArjoHuntleigh equipment.

Your *MaxiSlide* is part of a series of quality products designed especially for hospitals, nursing homes and other health care uses.

We are dedicated to serving your needs and providing the best products available along with training that will bring your staff maximum benefit from every ArjoHuntleigh product.

Please contact us if you have any questions about the operation or maintenance of your ArjoHuntleigh equipment.

## Conditions

- The unit is cared for in accordance with recommended, published “Instructions for Use”.
- The product care, in accordance with ArjoHuntleigh requirements, must begin on first use of the unit by the customer.
- The equipment is used for its intended purpose only and is operated within the published limitations.

- Only ArjoHuntleigh designated spare parts should be used.

Before using your *MaxiSlide* please read this manual thoroughly in its entirety. Information in the manual is crucial to the proper operation and maintenance of the equipment, and will help protect your product and ensure that the equipment performs to your satisfaction. Some of the information in this booklet is important for your safety and must be read and understood to help prevent possible injury. If there is anything in the manual that is confusing or difficult to understand, please call ArjoHuntleigh or their appointed distributor (the telephone number appears on the last page of this manual).

The following techniques are only a sample of the latest ergonomically correct patient handling manoeuvres carried out with the *MaxiSlide*.

# Safety Instructions

---

Symbols used adjacent to the text in these instructions:-

**Warning:** Means:- failure to understand and obey this warning may result in injury to you or to others.

- **Note:** Means:- this is important information for the correct use of this system or equipment.

See also “Care of your *MaxiSlide*” section.

The techniques shown are based on sound biomechanical principles. Pushing actions are carried out at a high level to avoid stooping and to allow use of the strong leg muscles. Pulling is carried out at a lower height, with the elbows straight, to avoid overuse of the shoulders and allow use of the leg muscles to achieve the movement. Where there is a choice it is preferable to push rather than pull.

Reference Mital:

- *Pushing 80 -120cm off ground*
- *Pulling 60-80cm off ground (depending on your height)*

Peak load is reduced by 50% if the manoeuvre is carried out slowly. This reduces the physical load experienced by the carer, and gentle movement is preferred by the patient.

Sliding sheets are generally removed once the manoeuvre is complete, but in some circumstances e.g. at night to allow patients more independent movement, they may be left in place. In these circumstances a suitable assessment must be carried out to ensure the patients safety.

**Warning:** Ensure that sliding sheets are never dropped or left on the floor, as they are extremely slippery and accidents could occur.

Full laundering instructions are included with the *MaxiSlide* but always ensure you follow your organizations infection control policy regarding laundering, disinfection and reuse of this product.

- **Note:** *Sliding techniques may be used successfully with patients of all dependency levels, but they must be subject to a suitable assessment. Factors you should consider are the environment, (bed, space, type of sliding sheets) characteristics of the patient (ability to assist, size, medical condition, comprehension, level of any pain, weight) the task you are carrying out, the carers individual capability (never attempt to carry out a task which you feel exceeds your individual capability).*

# Using your *MaxiSlide*

## General

The *MaxiSlide* is a unique, ergonomically designed device having ultra-low friction properties, which will aid carers in moving or positioning patients whilst, at the same time, encouraging good posture without causing undue stress to either the patient or the carer.

The *MaxiSlide* will also encourage the patient to help themselves wherever possible.

**Warning:** *MaxiSlide* has not been designed to, and must not therefore be used to lift patients.

This document uses the following ArjoHuntleigh patient categories from the 'Residents Gallery':

*Albert:* is able to perform daily activities independently, but may require special aids.

*Barbara:* is capable of performing most daily activities independently, with some assistance.

*Carl:* requires assistance with many of his daily activities.

*Doris:* requires a high level of assistance with all her daily activities.

*Emma:* is not capable of performing daily activities independently.

## *Carl, Doris & Emma - Fitting two MaxiSlides using the unravelling technique*

Some patients may not be suitable for rolling, for example when they are in pain, or due to their size. In this instance this technique may be used to insert two full length *MaxiSlide* sheets. The technique is firstly shown as a detail without the patient so it is easier to see what is happening.

Place both *MaxiSlide* sheets together lengthwise. Fold the two *MaxiSlide* sheets into 6"-8" (152 – 203mm) folds and then turn over so that the folds are underneath.

Hold both *MaxiSlide* sheets with your outside hand to secure them, and with the inside hand palm up unravel each fold, taking care to keep your hand in contact with the bed, do not lift upwards. Unravelling the folds of the sheets away from the patient prevents skin shear.

# Using your *MaxiSlide*

---

It is recommended that you unravel the *MaxiSlide* sheets from the patients head downwards, as this follows the direction of the patients clothes and helps prevent the sheets getting caught.

In case of pressure relieving mattresses gently push down on the mattress to assist with fitting. You may also find it helpful to set the mattress pressure to a firm setting for the procedure.

## *Doris - Turning in bed with two people*

This technique can be used with a patient who is unable to assist and hence requires 2 carers to reposition in bed.

Ensure the bed is at the correct height for the fitting of the *MaxiSlide* sheets and using the 'unravelling technique' fit them underneath the patient, starting at the patient's head.

Using the principles of normal movement wherever possible, prepare the patient to turn to the side desired.

The patient is moved towards one side of the bed to allow space to turn onto her side. At this point the bed is lowered, as the physical action to be undertaken is mainly pulling.

The carer grasps the *MaxiSlide* handles and adopts a 'walk' stance and using her bodyweight pulls the patient gently towards her.

Using the principles of normal movement the carer then turns the patient towards the second carer using a gentle rolling movement. The patient may need to have her hips, shoulders and legs repositioned for comfort.

Remove both *MaxiSlide* sheets from behind the patient rather than the front, as this is where the majority of the slide material ends up following the technique.

# Using your *MaxiSlide*

## *Doris - Moving up the bed, two people, pulling*

Use of a profiling bed can prevent a patient from slipping down towards the foot end. However, if such a bed is not available the following technique can be used to slide the patient back towards the head end

This technique may be used where the bed is at a low height, hence a pulling action is desirable. The patient is unable to assist with the movement.

The *MaxiSlide* sheets and tube have been fitted beneath the patient to remove any 'shearing' effects of the movement.

Each carer stands either side of the bed and holds the top *MaxiSlide* with the closest hand to the bed.

Ensuring good communication the carers move simultaneously and slowly backwards to the head of the bed.

**Warning:** Care must be taken that the pulling action is not carried out too quickly as this may result in injury to the patient.

## *Emma - Lateral transfer onto trolley*

Two *MaxiSlide* sheets are fitted using the 'unravelling' method. Use a Maxitube under the patient's feet if necessary.

- Note:** It is vital that you ensure that the trolley and the bed are of approximately equal height. It will however assist with the manoeuvre if the surface the patient is being transferred onto is slightly lower than the bed.

Ensure the bed and trolley castors are solidly locked.

Four Pull Straps are fitted to the top *MaxiSlide*, at shoulders, hip and feet. The Pull Straps prevent the carers from having to over reach.

The trolley is brought into place.

While the third carer rolls or tilts the patient towards him the two carers place a 'pat slide' on the bed and slide it underneath the *MaxiSlide* sheets.

# Using your *MaxiSlide*

The two carers on the pulling side grasp the Pull Straps. The carer on the 'pushing' side will assist with initiating the transfer, taking care not to overstretch.

Using clear commands and working as a team the two carers use a 'walk' stance and step back from the trolley, using their bodyweight to move the patient.

Final repositioning in the center of the trolley is achieved by using the handles on the *MaxiSlide*.

Remove both slide sheets to complete the move.

## *Moving out of a confined space on the floor (any fallen patient)*

On finding a patient on the floor always ensure the correct health checks have been made prior to attempting to move them. If the patient is assessed as safe to move, this technique can be used to manoeuvre them into a space where a hoist can be used.

Place a pillow under the patient's head.

Using two *MaxiSlide* sheets and the 'unravelling' technique place the sheets beneath the patient, ensuring they have been positioned beneath the pillow, this will protect the patient's head during the movement.

If space allows attaching the Pull Straps will enable you to stand, and work in a healthy posture.

If you require moving the patient over a greater distance you can reposition the lower *MaxiSlide* in front of the direction of travel.

# Care of your *MaxiSlide*

---

## *Cleaning Instructions*

The *MaxiSlide*, *MaxiTube*<sup>™</sup>, *MaxiTransfer*<sup>™</sup> can be wiped clean using either a chlorine based solution (up to 10,000 ppm), or an alcohol based solution (up to 70%).

In addition they can be washed at a maximum temperature of 80°C (175°F), and drip dried.

If necessary a low temperature tumble dry can reduce drying times however, the sliding sheets may develop a wrinkled appearance which should not affect performance.

## *Attention!*

- *Please ensure that the MaxiSlide, MaxiTube, MaxiTransfer are always packed away or hung up after use in order to prevent people accidentally slipping on them.*
- *The MaxiSlide, MaxiTube, MaxiTransfer are not designed to be left directly under the patient. However in the case where there is a need for long term patient positioning in bed, it may be possible, based on a clinical evaluation of the patient and using an interface such as a cotton sheet, to leave the product under the patient*
- *Please check the MaxiSlide, MaxiTube, MaxiTransfer regularly for any signs of wear and replace as necessary.*
- *The MaxiSlide, MaxiTube, MaxiTransfer should only be used where no better alternative to moving or positioning the patient is available.*

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**NORGE**

ArjoHuntleigh Norway AS  
Olaf Helsets vei 5  
N-0694 OSLO  
Tel: +47 22 08 00 50  
Faks: +47 22 08 00 51  
E-mail:  
no.kundeservice@arjohuntleigh.com

**ÖSTERREICH**

ArjoHuntleigh GmbH  
Dörrstrasse 85  
AT-6020 INNSBRUCK  
Tel: +43 (0) 512 204 160 0  
Fax: +43 (0) 512 204 160 75

**POLSKA**

ArjoHuntleigh Polska Sp. z o.o.  
ul. Ks Piotra Wawrzyniaka 2  
PL-62-052 KOMORNIKI (Poznan)  
Tel: +48 61 662 15 50  
Fax: +48 61 662 15 90  
E-mail: arjo@arjohuntleigh.com

**PORTUGAL**

ArjoHuntleigh em Portugal  
MAQUET Portugal, Lda. (Distribuidor Exclusivo)  
Rua Poeta Bocage n.º 2 - 2G  
PT-1600-233 Lisboa  
Tel: +351 214 189 815  
Fax: +351 214 177 413  
E-mail: Portugal@arjohuntleigh.com

**SUISSE / SCHWEIZ**

ArjoHuntleigh AG  
Fabrikstrasse 8  
Postfach  
CH-4614 HÄGENDORF  
Tél/Tel: +41 (0) 61 337 97 77  
Fax: +41 (0) 61 311 97 42

**SUOMI**

Oy Vestek AB  
Martinkuja 4  
FI-02270 ESPOO  
Puh: +358 9 8870 120  
E-mail: info@vestek.fi

**SVERIGE**

ARJO Scandinavia AB  
Hans Michelsensgatan 10  
SE-211 20 MALMÖ  
Tel: +46 (0) 10 494 7760  
Fax: +46 (0) 10 494 7761  
E-mail: kundservice@arjohuntleigh.com

**UNITED KINGDOM**

ArjoHuntleigh UK  
ArjoHuntleigh House  
Houghton Hall Park  
Houghton Regis  
UK-DUNSTABLE LU5 5XF  
Tel: +44 (0) 1582 745 700  
Fax: +44 (0) 1582 745 745  
E-mail: sales.admin@arjohuntleigh.com

**USA**

ArjoHuntleigh Inc.  
2349 W Lake Street Suite 250  
US-Addison, IL 60101  
Tel: +1 630 307 2756  
Free: +1 800 323 1245 Institutional  
Free: +1 800 868 0441 Home Care  
Fax: +1 630 307 6195  
E-mail: us.info@arjohuntleigh.com

# ARJOHUNTLEIGH


GETINGE GROUP

[www.arjohuntleigh.com](http://www.arjohuntleigh.com)

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö  
SWEDEN

**GETINGE GROUP** is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **GETINGE** and **MAQUET**. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.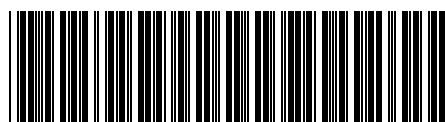

NSX01000 - INT4