# homecraft Ascot Combined Board and Seat System

|          | CODE          | 09 CODE     | US CODE     |
|----------|---------------|-------------|-------------|
| Standard | <b>AA1100</b> | 09 107 2701 |             |
| Cut-away | <b>AA1104</b> | 09 107 2735 | 08 130 5028 |

## Intended uses

- To compensate for a disability by helping a person get into and out of a bathtub and by providing a stable sitting surface

## Instructions for installation

1. Remove the bath board (labelled **A** in Figure 1) from the seat (**B**) by loosening the seat height adjuster knobs (**C**) and pulling the securing legs (**D**) out of the seat height adjusters.
2. Position the seat in the bathtub and put the securing legs of the board through both parts of the seat height adjusters.
3. Allow the board to slide down until it rests on both bathtub side rims. Tighten the seat adjuster knobs to secure the board at the appropriate height.
4. Loosen both wing nuts on each side adjuster bracket of the board (**E**) and slide the brackets out until they fit snug against the sides of the bathtub. Tighten the four wing nuts securely.
5. A properly positioned board and seat system is shown in Figure 2.

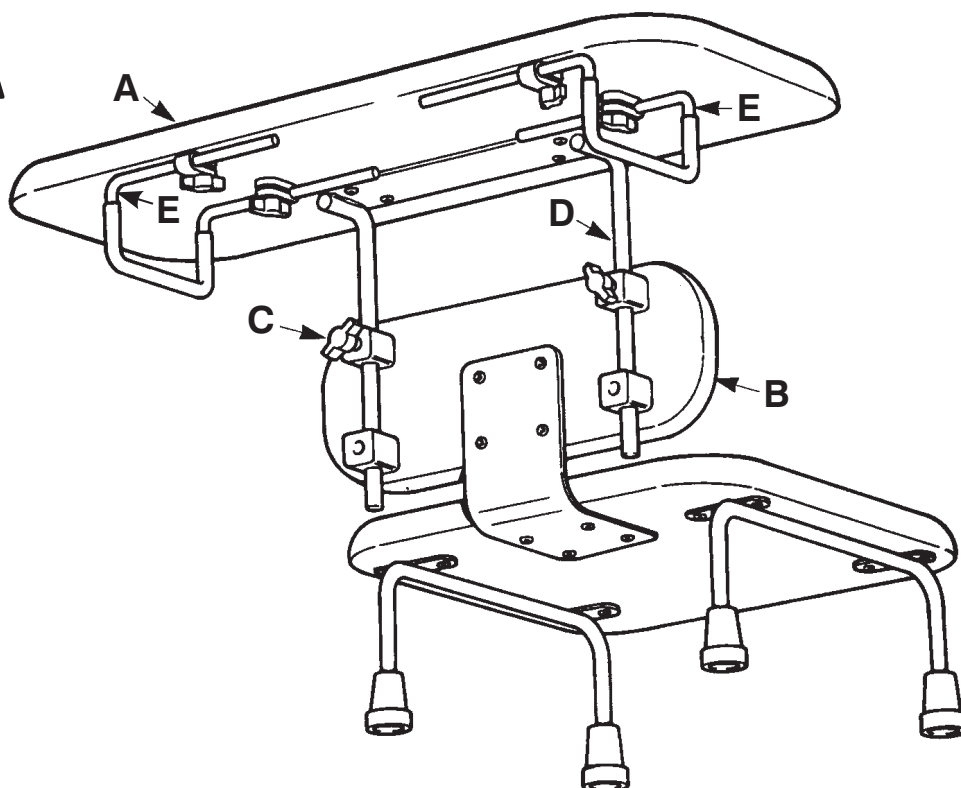
## Care and Cleaning

The board and seat system can be cleaned with warm water and mild detergent or household disinfectant.

## Cautions

- The board and seat system is not to be used to support more than 127 kg / 280 lb / 20 st.
- Be sure the board and seat system is securely in place before putting weight on it.
- Cover the board and seat with a towel to help reduce slipperiness.

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