

# Nighthawk Manufacturing

## 02 Holder Installation

PART # 3201

### Universal Wheelchair D and E Cylinder Metal Oxygen Holder



- Holds D and E cylinder oxygen bottles
- 13 1/2" tall and 4 1/2" inside diameter
- Oxygen bottles are secured with a thumb screw for easy removal
- Comes with 1" clamps that can reduce to 7/8" with included sleeve
- Top mounting bar slides up and down plus both mounting clamps swivel to mount to just about any manual wheelchair and many electric wheelchairs (spacer sleeves included for clamps for use on smaller diameter tubing)
- Once properly mounted, oxygen bottles are secure and won't move around like fabric holders will
- Black finish and solid metal construction for lasting durability



### Top Clamp



- Clamp to top pipe under armrest (If pipe too small use included spacers) tighten screw partially



### Bottom Clamp



- Then Clamp to Bottom pipe where anti-tipper is (If pipe is too small use included spacer)
- then tighten screws and adjust till O2 Holder is straight then finish tightening screws

