

Molift Raiser

TRANSFER PLATFORM FOR ACTIVE TRANSFER



Safe and easy to use

Molift Raiser platform is designed to safely move the user from a sitting to standing position while maintaining the best possible ergonomic moving and handling conditions for the caregiver. The unique leg support is designed to be easily adjusted with one hand. Similarly, wheel locks (right or left) will lock both wheels simultaneously providing a safe braking solution in space limited environments. The fine grain finish of the footplate provides a non-slip surface and the handle offers many different grip possibilities. These features enable the user to feel safe throughout the entire transfer.

Molift Raiser has several unique features that makes the transfer safe and easy.

Molift	Cm	Inch	Item no.
Raiser	58 x 63 x 120	22.8 x 24.8 x 47.2	16090202

Material: Steel, plastic, aluminium, thermoplastic rubber

Total Weight: 14.2 kg (31.3 lbs)



Cleaning instruction

Use a solvent-free detergent with a pH level between 5 and 9. Can be washed at 80° C for 3 minutes (not with leg support). Use a wet wipe for the leg support. Do not rinse directly on bearings and leg support with a pressure washer.



Features



Leg support with one-hand height regulation

The soft padded leg support provides comfortable support for the user. It is simple to adjust the height with only one hand.



Handle with many grip possibilities

The handle provides many grip possibilities for both caregivers and users.



One foot locks both wheels

The foot brake locks both sides whether you choose the left or right one. This ensures a smooth and safer use of the product.



Low boarding

The low step-on height makes it easier for users to place their feet on the platform.



If the user has difficulty reaching the handle, the Soft Handles can be used to help the user reach all the way.



The Raiser Strap can provide support around the back or hips when standing up.

The user can be assisted by one or two caregivers using the Raiser Strap+.

Accessories



Soft handles

If the user has difficulty reaching the handle, the Soft Handles can be used to help the user reach all the way.

Item no.

82511



Safety strap with sliding function – Raiser Strap+

When users need help to be assisted, this can be done by one or two caregivers using the Raiser Strap+.

S/M Item no.

82528

L/XL Item no.

82529



Safety strap – Raiser Strap

Provides support around the back or hips when standing up.

S/M Item no.

82508

L/XL Item no.

82509