

# Quick Start Instructions

Need Set-up Help? (800) 650-3637

## 1 Mount Door Monitor to the Wall

- A** Peel off yellow backing from adhesives (3) on the back of the door monitor. The door bar should be mounted to clean, smooth surfaces.


Door Monitor



- B** Adhere the door bar vertically centered to the outside of either door frame with cords facing up or down (in direction of power source).

*Warning: If mounting door monitor near metal frame, mount at least 8" away from metal frame.*

- C.** Mount the Reset Button to wall using the provided double-sided tape, up to 40 feet away.

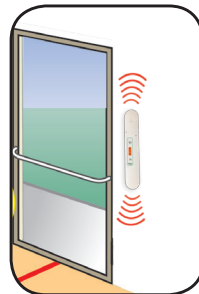

## 2 Plug in the AC Adapter

Door Monitor  
To Optional  
Magnetic Contact  
Switch

Door Monitor

- A** Plug in AC Adapter to female plug attached to the door alarm

- B** Plug in AC Adapter to wall outlet

AC Outlet

AC Adapter

*Note: Power light will illuminate to indicate system is functioning.*

## 3 Test the System


- A** Test the range of the door monitor by approaching the door with a resident transmitter from all directions, making sure that the monitor sound goes off at desired distances.

- B** To silence the alarm, push the wall mount reset button.

- C** Wait approx. 20 seconds until **Reset/Pause** light goes off before the system is ready.

- D** Attach transmitter to resident's wrist with the included strap.


*Warning: Door monitor will sound if door is opened and a resident wearing a transmitter is within 8-14 feet from the door monitor. If the alert is triggered when a transmitter is too close or too far away from door, or if you can pass through door, call for technical assistance.*

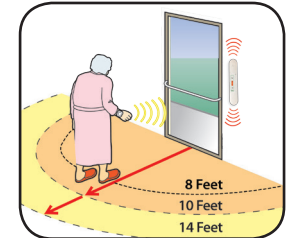


## 4 (Optional) Mount Magnetic Contact Switch

*WHEN USING THE MAGNETIC SWITCH, THE ALARM WILL ONLY SOUND WHEN DOOR IS OPENED AND IF A RESIDENT IS WEARING A WRISTBAND WITHIN THE MONITORED FIELD.*

- A** Screw magnetic contact (see below) to underside of door frame with wording facing open door. Lead cord along outside of door frame.

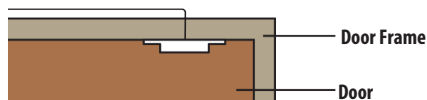

- B** Screw magnet (see below) to door with arrow facing up.

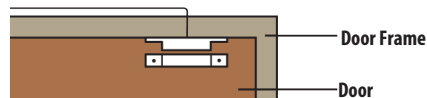

- C** Magnetic contact and magnet must align perfectly to function.

- D** Plug magnetic contact switch to male plug attached to door alarm.

## 5 Test the System with Magnetic Contact Switch

- A** Test the monitor with the magnetic contact switch installed by opening the door with an activated resident transmitter in hand. Door monitor will sound when door is opened.

- B** To silence the alarm, push the wall mount reset button.

- C** Wait approximately 20 seconds until **Reset/Pause** light goes off before the system is ready.

- D** Attach transmitter to resident's wrist with the included strap.

*Warning: Door monitor will sound if door is opened and a resident wearing a transmitter is within 8-14 feet from the door monitor. If the alert is triggered when a transmitter is too close or too far away from door, or if you can pass through door, call for technical assistance.*

## Troubleshooting

Call Smart Caregiver for assistance at (800) 650-3637