

I N S T R U C T I O N S

Bedside Beverage Holder

Catalog Number 745760002

1. Open the clamp and place it around frame of the bed or wheelchair
2. Slide the threaded rod at the end of the gooseneck through the clamp
3. Screw the lever knob onto the rod making sure the cupholder is in an upright position
4. Tighten the lever knob

For frames under 1" in diameter, a plastic reducer is supplied. Position the reducer on the frame and put the clamp over it.

Caution: Hot beverages should not be placed in the beverage holder.

Caution: A container holding more than 20 oz. of fluid should not be placed in the beverage holder.



A Division of SP Industries

661 Route 23 South • Wayne, NJ 07470

973-628-7600 • email custservice@maddak.com • www.spableware.com

974576201

0616

I N S T R U C T I O N S

Bedside Beverage Holder

Catalog Number 745760002

1. Open the clamp and place it around frame of the bed or wheelchair
2. Slide the threaded rod at the end of the gooseneck through the clamp
3. Screw the lever knob onto the rod making sure the cupholder is in an upright position
4. Tighten the lever knob

For frames under 1" in diameter, a plastic reducer is supplied. Position the reducer on the frame and put the clamp over it.

Caution: Hot beverages should not be placed in the beverage holder.

Caution: A container holding more than 20 oz. of fluid should not be placed in the beverage holder.



A Division of SP Industries

661 Route 23 South • Wayne, NJ 07470

973-628-7600 • email custservice@maddak.com • www.spableware.com

974576201

0616