



Etac Clean

Shower commode chairs

 **etac**®

Etac Clean

Etac Clean is secure, safe and simple for both users and carers. With practical functionality and design, every detail of the chair has been developed to give the user a positive and safe bathroom experience.

Easy to fit and manoeuvre

Etac Clean is extremely easy to operate, thanks to the width of the push handle. The clever design can be seen throughout the entire chair. The upright handle ensures that Clean is a perfect fit, even over wall-mounted toilets, and the chair is open at the back to facilitate access.

Comfortable and practical foot support

The foot support is unique and can be pushed under the seat when not in use.

The soft curved surface provides good comfort for the feet, which are also protected by the foot support's side protection.

All other variants of Clean also have these features, but with additional functionality to meet other needs.



Etac Clean shower commode chair

44 cm (17¼"), 4 lockable castors

49 cm (19¼"), 4 lockable castors

49 cm (19¼"), 2 lockable castors

49 cm (19¼"), 4 lockable wheels, backrest upholstery, panholder

49 cm (19¼"), 2 lockable castors, soft back

55 cm (21¾"), 4 lockable wheels, backrest upholstery, panholder

55 cm (21¾"), 4 lockable castors

55 cm (21¾"), 2 lockable castors

55 cm (21¾"), 2 lockable castors, soft back

60 cm (23½"), 2 lockable castors incl. heightening spacers

White

80229269

80229270

80229273

80229286

80229305

80229287

80229271

80229274

80229306

80229275

Lagoon green

80229209

80229210

80229213

-

-

-

80229211

80229214

-

80229215

Etac Clean shower commode chair - double box

49 cm (19¼"), 4 lockable castors, double box

49 cm (19¼"), 2 lockable castors, double box

55 cm (21¾"), 4 lockable castors, double box

55 cm (21¾"), 2 lockable castors, double box

White

-

80229283

80229281

80229284

Lagoon green

80229220

80229223

80229221

80229224

Weight

44 cm: 13.5 kg

49 cm: 14.0 kg

55 cm: 14.5 kg

60 cm: 15 kg

Maximum user weight

130 kg

Facts



Material

Chassis: Powder-painted steel tubing

Seat/back support: Polypropylene

Arm supports: Polypropylene

Foot support: Polypropylene

Castor brackets: Polyamide

Castors, lockable, ball-bearing: Polyamide/TPE

Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.
The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

Features



Slim but roomy

The smart design gives Etac Clean a large and comfortable seating area even though the chair is neat and slim.



Fits over the toilet

Thanks to the upright operating handle, the chair is a perfect fit, even over wall-mounted toilets.




Easy access

The chair is open at the back to facilitate access for hygiene purposes.



Foot support designed for feet

The practical foot support has a soft, rounded shape which gives the feet extra comfort and protection.

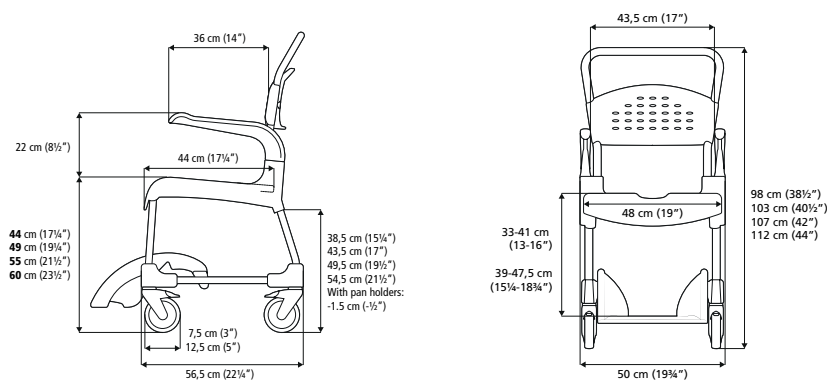



Comfort and support

The soft enveloping seat and back support provide stability, warmth and softness.

Integrity in exposed situations

Clean Comfort is designed to increase the feeling of integrity and dignity. The comfort padding protects and hides the user without reducing accessibility for the carers.


Etac Clean Comfort

All the features of Etac Clean can also be found in this chair, but with even greater comfort. The seat, arm support and back support are padded and guarantee soft and comfortable seating.

Soft and warmth

Etac Clean Comfort enhances the feeling of well-being for those who need a little extra comfort and support in the bathroom. The water-repellent and heat-reflective cover of the back support and arm supports helps the user to stay warm. The contoured seat offers comfortable support for the thighs and envelops the user for a safer and more comfortable feeling.

Etac Clean Comfort has the same basic benefits as other members of the Clean family
– but now with even greater comfort.

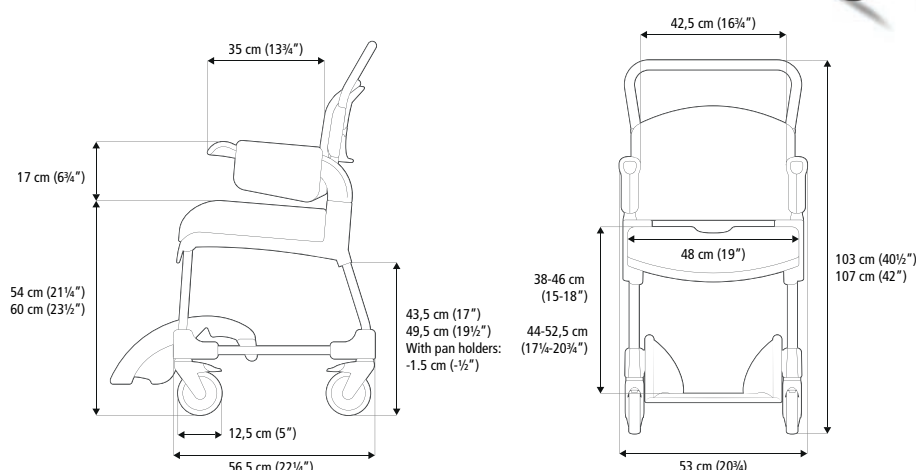

Etac Clean Comfort

9 cm (19 1/4"), incl. Comfort Cover, Comfort Seat Soft, 4 lockable castors, white
55 cm (21 3/4"), incl. Comfort Cover, Comfort Seat Soft, 4 lockable castors, white

Item no.

80229298

80229299

Material

Comfort Cover: Polyethylene
Comfort Seat Soft: Polyurethane
Chassis: Powder-painted steel tubing
Seat / back support: Polypropylene
Arm supports: Polypropylene
Foot support: Polypropylene
Castor brackets: Polyamide
Castors, lockable, ball-bearing: Polyamide/TPE

Opening: depth x width

36.9 x 18.2 cm

Weight

49 cm: 13.5 kg

55 cm: 14 kg

Maximum user weight

130 kg

Maintenance



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.
The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.



The same chair for all heights

The smart design means that the chair is a perfect fit above all wall-mounted toilets, regardless of height.

Additional support for the legs

In the event of additional need for leg and foot support, various accessories are available, e.g. split foot supports and calf supports.


Etac Clean Height Adjustable

In addition to all the features of the basic model, this variant allows you to adjust the height of the seat.

Adjust the height for a perfect fit

The possibility of adjusting the seat height means that Etac Clean Height Adjustable can be tailor-made to suit the user's needs – for example to make transfers easier or to fit perfectly over the toilet. Adjustment is simple and no tools are required.

Etac Clean Height Adjustable has the same basic benefits as the other members of the Clean family – but is also height-adjustable up to 60 cm, and has a greater range of foot supports and calf supports.

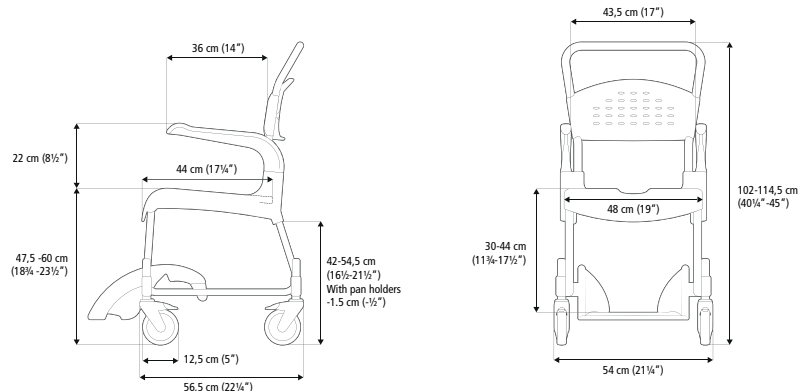

Etac Clean Height Adjustable

4 lockable wheels, white

Item no.

80229208

Maintenance



Weight

14.2 kg

Maximum user weight

130 kg

Material

Chassis: Powder coated steel tubing
 Seat/Back support: Polypropylene
 Arm supports: Polypropylene
 Foot support: Polypropylene
 Castors with ball, bearings: Polyamide/TPE/Stainless steel tubing
 Castor height adjustment tubes: Stainless steel

Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.
 The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.



Self propelled

Easy to operate for the independent user.

Safe and simple transfer

The flat seat and the brake position facilitate lateral transfer.


Etac Clean 24"

This variant of Etac Clean is suitable for those who can operate the chair independently and want to maximise their mobility.

Easy to manoeuvre, for the independent user

The Etac Clean 24-inch wheels are positioned to make it easy to manoeuvre the chair without compromising user safety or comfort. It is easy to transfer to and from the chair thanks to the large, unrestricted seat in front of the wheels. The brake handles are also positioned below the seat so that they do not get in the way during transfer, while remaining easy to reach.

Etac Clean 24" has the same basic benefits as the other members of the Clean family
– but allows the user to operate the chair independently.

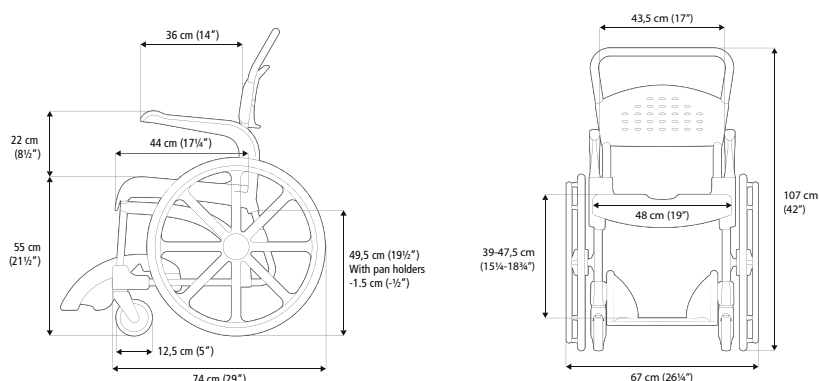

Etac Clean 24"

55 cm (21 1/4") with 24" rear wheels, white

55 cm (21 1/4") with 24" rear wheels, lagoon green

Item no.

80229276

80229216

Material

Rear wheel adapter: aluminum
Rear wheels: polyamide, solid tires
Chassis: Powder-painted steel tubing
Seat/back support: Polypropylene
Arm supports: Polypropylene
Foot support: Polypropylene
Caster brackets: Polyamide
Castors, ball-bearing: Polyamide/TPE

Maintenance



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.
The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

Accessories



Soft back support

80209225
Adjustable.
Washing at max 60° C (140° F).



Side Support

80209509
Adjustable in width. Material: Polyester fabric, Velcro, polyethylene. Washing at max. 60° C (140° F). *Can not be used in combination with: 80209225.*




Crossbar

80209246
Attached to the arm supports. Material: Aluminium, cellular polyethylene, polypropylene. *Can not be used in combination with: 80209228, 80209237 and 80209226.*



Chest harness strap

0209442
Adjustable in length. Attach to soft back support. Material: Polyester.

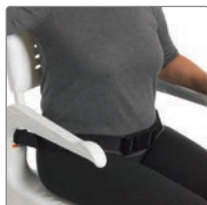

Positioning belt.

80209440
Adjustable in length. Attachment with velcro straps. Material: Polyester.



Safety strap

80209034
Attachments on the rear side of the back support. Always use together with positioning belt. Material: Polyester. *Can not be used in combination with: 80209225.*



Cushion, protect

80209245
Air-inflated ROHO cushion with excellent pressure distribution properties. Adjustable air pressure. Recess width: 21 cm (8¼"). Material: Neoprene. Cleaning max: 60° C (140° F).



Soft pad

Grey: 80209260
Heat-reflecting with a non-slip surface. Recess width: 17 cm (6¾"). Material: polyethylene. Cleaning max max. 60° C (140° F) or with alcohol wipe.



Soft back pad

Grey: 80209261
Heat-reflecting with a non-slip surface. Material: polyethylene. Cleaning max. 60° C (140° F) or with alcohol wipe.



Seat cover

80209247
When a complete seat is required. Material: Polyurethane. Cleaning max. 85° C (185° F) or with alcohol wipe.



Comfort Kit

80209237
Includes both Comfort Seat Soft and Comfort Cover. Cleaning max. 60° C (140° F) or with alcohol wipe. *Can not be used in combination with: 80209225 and 80209246.*



Comfort Cover

80209228
A soft back support with integrated arm support pads. Material: polyethylene. Cleaning max. 60° C (140° F) or with alcohol wipe. *Can not be used in combination with: 80209225 and 80209246.*



Comfort Arm Pads

80209226
Soft and warm, leaving the handles free to ensure a firm grip when standing up. Material: polyethylene. Cleaning max. 60° C (140° F) or with alcohol wipe. *Can not be used in combination with: 80209246.*



Comfort Seat Soft

Small opening width 15 cm (6"): . 80209301
narrow opening width 18 cm (7"): 80209227
A soft contoured seat. Material: polyurethane. Cleaning max. 85° C (185° F) or with alcohol wipe.

Accessories



Soap dish (does not fit Clean 24")

Grey: 80209266
Attaches to the side of the seat. Built-in shower handle holder.
Material: Polypropylene.
(Does not fit Clean 24")



Pan

Grey: 80209254
Material: polypropylene.
Pan incl. lid with handle
Grey: 80209255
Material: Polypropylene.



Pan holder

Grey: 80209257
Fits both pan, bedpan and regular hospital pans with a thin rim.



Bucket with handle and lid

80302026
The bucket holder is not included.
Material: Polypropylene.



Bucket holder

Grey: 80209258
Fits Bucket with handle and lid and other buckets with a thick rim. Material: Polypropylene.



Attachment for low mounting of pan/bucket

Grey: 80209259
Fitted to the pan/bucket holder. Lowers the pan/bucket holder with 6 cm (2½").
Material: Polypropylene.



Disposable pans

80209282
Strong and firm. 50 pcs.



Disposable pan holder kit

Grey: 80209229
Fits Disposable pan.

Additional accessories – Clean Height Adjustable only



Soap dish (does not fit Clean 24")

Grey: 80209266
Attaches to the side of the seat. Built-in shower handle holder.
Material: Polypropylene.
(Does not fit Clean 24")



Pan

Grey: 80209254
Material: polypropylene.
Pan incl. lid with handle
Grey: 80209255
Material: Polypropylene.

Additional accessories – Clean 49 and 55 cm only



Attachment for low mounting of pan/bucket

Grey: 80209259
Fitted to the pan/bucket holder. Lowers the pan/bucket holder with 6 cm (2½").
Material: Polypropylene.



Disposable pans

80209282
Strong and firm. 50 pcs.



75 mm (3") castors

White, front, pair: 80209515
White, back pair: 80209516
Lagoon green, front, pair: 80209513
Lagoon green, back, pair: 80209514
Reduce seat height by 5 cm (2").
Mounted on Clean 44 cm as standard.
Material: polyamide/TPE.