MAXISLIDE™

ARJOHUNTLEIGH
GETINGE GROUP

GB
2-9

CA/FR
10-17

CA/US
18-25

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Some of the information contained in these instructions may become outdated, due to improvements made to the product in the future. If you have any questions regarding these instructions or your *MaxiSlide*™, please contact ARJO or their approved distributor.

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Our policy is one of continuous development and we therefore reserve the right to make technical alterations without notice. The content of this publication may not be copied either whole or in part without the consent of ARJO Med. AB LTD.

ARJO Med. AB LTD. strongly advise and warn that only ARJO Company Designated Parts, which are designed for the purpose, should be used on equipment and other appliances supplied by ARJO, to avoid injuries attributable to the use of inadequate parts

Un-authorized modifications on any ARJO Med. AB LTD. equipment may affect it's safety and are in breach of any warranty on it. ARJO Med. AB LTD. will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any un-authorized modification to it's products.

The ARJO Company's Conditions of sale make specific provision confirming no liability in such circumstances.

Contents

SECTION	Page No.
Forward	3
Safety Instructions	3
Using your <i>MaxiSlide</i>	4-8
General	4
Carl, Doris & Emma - Fitting 2 <i>MaxiSlides</i> using the unravelling technique	4
Doris - Turning in bed with two people	5
Doris-Movingupthebed,twopeople,pulling	6
Emma - Lateral transfer onto trolley	6
Moving out of a confined space on the floor	7
Care of your <i>MaxiSlide</i>	8
Ordering Details	9

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Your *MaxiSlide*™ is part of a series of quality products designed especially for hospitals, nursing homes and other health care uses.

We are dedicated to serving your needs and providing the best products available along with training that will bring your staff maximum benefit from every ARJO product.

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Conditions

- The unit is cared for in accordance with recommended, published “Operating and Product Care Instructions”.
- The product care, in accordance with ARJO requirements, must begin on first use of the unit by the customer.
- The equipment is used for its intended purpose only and is operated within the published limitations.

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Before using your *MaxiSlide*™, please read this manual thoroughly in its entirety. Information in the manual is crucial to the proper operation and maintenance of the equipment, and will help protect your product and ensure that the equipment performs to your satisfaction. Some of the information in this booklet is important for your safety and must be read and understood to help prevent possible injury. If there is anything in the manual that is confusing or difficult to understand, please call ARJO Med. AB. Ltd or their appointed distributor (the telephone number appears on the last page of this manual).

The following techniques are only a sample of the latest ergonomically correct patient handling manoeuvres carried out with the *MaxiSlide*™. There are many more shown in our CD-ROM *MaxiSlide*™ user guide which is available from your local ARJO office.

Safety Instructions

Symbols used adjacent to the text in these instructions:-

Warning: Means:- failure to understand and obey this warning may result in injury to you or to others.

- **Note:** Means:- this is important information for the correct use of this system or equipment.

See also “Care of your *MaxiSlide*™” section.

The techniques shown are based on sound biomechanical principles. Pushing actions are carried out at a high level to avoid stooping and to allow use of the strong leg muscles. Pulling is carried out at a lower height, with the elbows straight, to avoid overuse of the shoulders and allow use of the leg muscles to achieve the movement. Where there is a choice it is preferable to push rather than pull.

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Peak load is reduced by 50% if the manoeuvre is carried out slowly. This reduces the physical load experienced by the carer, and gentle movement is preferred by the patient.

Sliding sheets are generally removed once the manoeuvre is complete, but in some circumstances e.g. at night to allow patients more independent movement, they may be left in place. In these circumstances a suitable assessment must be carried out to ensure the patients safety.

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Full laundering instructions are included with the *MaxiSlide*™ but always ensure you follow your organizations infection control policy regarding laundering, disinfection and reuse of this product.

- **Note:** Sliding techniques may be used successfully with patients of all dependency levels, but they must be subject to a suitable assessment. Factors you should consider are the environment, (bed, space, type of sliding sheets) characteristics of the patient (ability to assist, size, medical condition, comprehension, level of any pain, weight) the task you are carrying out, the carers individual capability (never attempt to carry out a task which you feel exceeds your individual capability).

Using your *MaxiSlide*

General

The *MaxiSlide*™ is a unique, ergonomically designed device having ultra-low friction properties, which will aid carers in moving or positioning patients whilst, at the same time, encouraging good posture without causing undue stress to either the patient or the carer.

The *MaxiSlide*™ will also encourage the patient to help themselves wherever possible.

Warning: *MaxiSlide*™ has not been designed to, and must not therefore be used to lift patients.

This document uses the following ARJO patient categories from the 'Residents Gallery':

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Albert and *Barbara* from the 'Residents Gallery' can be found on the *MaxiSlide*™ CD Rom User Guide.

Carl, Doris & Emma - Fitting 2 MaxiSlide™s using the unravelling technique

Some patients may not be suitable for rolling, for example when they are in pain, or due to their size. In this instance this technique may be used to insert two full length *MaxiSlide*™ sheets. The technique is firstly shown as a detail without the patient so it is easier to see what is happening.

Place both *MaxiSlide*™ sheets together lengthwise. Fold the two *MaxiSlide*™ sheets into 6-8" folds and then turn over so that the folds are underneath.

Hold both *MaxiSlide*™ sheets with your outside hand to secure them, and with the inside hand palm up unravel each fold, taking care to keep your hand in contact with the bed, do not lift upwards. Unravelling the folds of the sheets away from the patient prevents skin shear.

Using your *MaxiSlide*

It is recommended that you unravel the *MaxiSlide*™ sheets from the patients head downwards, as this follows the direction of the patients clothes and helps prevent the sheets getting caught.

In case of pressure relieving mattresses gently push down on the mattress to assist with fitting. You may also find it helpful to set the mattress pressure to a firm setting for the procedure.

Doris - Turning in bed with two people

This technique can be used with a patient who is unable to assist and hence requires 2 carers to reposition in bed.

Ensure the bed is at the correct height for the fitting of the *MaxiSlide*™ sheets and using the 'unravelling technique' fit them underneath the patient, starting at the patient's head.

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The patient is moved towards one side of the bed to allow space to turn onto her side. At this point the bed is lowered, as the physical action to be undertaken is mainly pulling.

The carer grasps the *MaxiSlide*™ handles and adopts a 'walk' stance and using her bodyweight pulls the patient gently towards her.

Using the principles of normal movement the carer then turns the patient towards the second carer using a gentle rolling movement. The patient may need to have her hips, shoulders and legs repositioned for comfort.

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Use of a profiling bed can prevent a patient from slipping down towards the foot end. However, if such a bed is not available the following technique can be used to slide the patient back towards the head end

This technique may be used where the bed is at a low height, hence a pulling action is desirable. The patient is unable to assist with the movement.

The *MaxiSlide*™ sheets and tube have been fitted beneath the patient to remove any 'shearing' effects of the movement.

Each carer stands either side of the bed and holds the top *MaxiSlide*™ with the closest hand to the bed.

Ensuring good communication the carers move simultaneously and slowly backwards to the head of the bed.

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Two *MaxiSlide*™ sheets are fitted using the 'unraveling' method. Use a Maxitube under the patient's feet if necessary.

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Four Pull Straps are fitted to the top *MaxiSlide*™, at shoulders, hip and feet. The Pull Straps prevent the carers from having to over reach.

The trolley is brought into place.

While the third carer rolls or tilts the patient towards him the two carers place a 'pat slide' on the bed and slide it underneath the *MaxiSlide*™ sheets.

Using your *MaxiSlide*

The two carers on the pulling side grasp the Pull Straps. The carer on the 'pushing' side will assist with initiating the transfer, taking care not to overstretch.

Using clear commands and working as a team the two carers use a 'walk' stance and step back from the trolley, using their bodyweight to move the patient.

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On finding a patient on the floor always ensure the correct health checks have been made prior to attempting to move them. If the patient is assessed as safe to move, this technique can be used to manoeuvre them into a space where a hoist can be used.

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Using two *MaxiSlide*™ sheets and the 'unravelling' technique place the sheets beneath the patient, ensuring they have been positioned beneath the pillow, this will protect the patient's head during the movement.

If space allows attaching the Pull Straps will enable you to stand, and work in a healthy posture.

If you require moving the patient over a greater distance you can reposition the lower *MaxiSlide*™ in front of the direction of travel.

Care of your *MaxiSlide*

Cleaning Instructions

The *MaxiSlide*[™], *MaxiTube*[™], *MaxiTransfer*[™] can be wiped clean using either a Chlorine based solution (up to 10000 ppm), an alcohol based solution (up to 70%) or any good quality sterilization solution.

In addition they can be washed at a maximum temperature of 80°C (175°F), and drip dried.

If necessary a low temperature tumble dry can reduce drying times however, the sliding sheets may develop a wrinkled appearance which should not affect performance.

Attention!

- *Please ensure that the MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] are always packed away or hung up after use in order to prevent people accidentally slipping on them.*
- *The MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] are not designed to be left directly under the patient. However in the case where there is a need for long term patient positioning in bed, it may be possible, based on a clinical evaluation of the patient and using an interface such as a cotton sheet, to leave the product under the patient*
- *Please check the MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] regularly for any signs of wear and replace as necessary.*
- *The MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] should only be used where no better alternative to moving or positioning the patient is available.*

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To Assist your staff further in other techniques of using your *MaxiSlide*TM, a CD-ROM or video is now available covering all the 'Residents Gallery' from *Albert* through to *Emma*. For further details, please contact your local ARJO sales representative.

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- *Introduction (Residents Gallery)*
- *Safe use of the MaxiSlide*TM
- *Fitting & removing the MaxiSlide*TM & *MaxiTube*TM - *Bridging and Scissors technique*
- *Independent moving up the bed - Knees bent, sitting position, using a trapeze, straight legs.*
- *Turning independently.*
- *Turning a patient, the hip tilt.*
- *Moving sideways and sitting on the edge of the bed.*
- *Getting into bed - independently and assisted.*
- *Fitting and removing full length MaxiSlide*TM
- *Unravelling technique*
- *Moving up the bed with assistance*
- *Turning a patient away*
- *Turning with one carer from one side of the bed*
- *Turning in bed with two people*
- *Moving towards the head of the bed with two carers - Pulling and pushing technique*
- *Turning into a prone position*
- *Lateral transfer onto a theatre and bath trolley*
- *Fitting a sling in a chair with a slide sheet*
- *Fitting a stretcher sling between two MaxiSlide*TM's
- *Fitting an X-Ray plate*
- *Moving out of a confined space on the floor (any fallen patient)*
- *Fitting a sling in a moulded chair using a MaxiTube*TM
- *Using a MaxiSlide*TM to remove sling on a bed

*Fitting a sling on a bed using MaxiSlide*TM's

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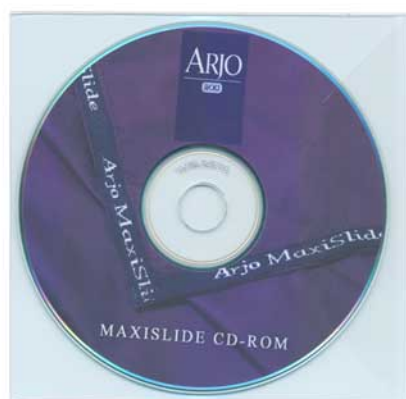
Introduction

The vertical and horizontal lines printed in the margins adjacent to the text/illustrations in these instructions are for ARJO use only and should be disregarded by the reader.

Some of the information contained in these instructions may become outdated, due to future product improvements. If you have any questions regarding these instructions or your *MaxiSlide™*, please contact ARJO or its approved distributor.

© ARJO Med. AB LTD

ARJO products are patented or patent pending. Patent information is available by contacting ARJO Med. AB LTD.

ARJO's policy is one of continuous development. ARJO therefore reserves the right to change specifications without notice. The content of this publication may not be copied either whole or in part without the consent of ARJO Med. AB LTD.

ARJO Med. AB LTD. strongly advises and warns that only ARJO Company Designated Parts, which are designed for the purpose, should be used on equipment and other appliances supplied by ARJO, to avoid injuries attributable to the use of inadequate parts

Unauthorized modifications on any ARJO Med. AB LTD. equipment may affect its safety and are in breach of any equipment warranty. ARJO Med. AB LTD. will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorized modifications to its products.

The ARJO Company's Conditions of Sale make specific provision confirming no liability in such circumstances.

Contents

SECTION	Page No.
Foreward	19
Safety Instructions	19
Using Your <i>MaxiSlide</i>	20-24
General	20
Carl, Doris & Emma - Fitting 2 <i>MaxiSlides</i> using the unravelling technique	20
Doris - Turning in bed by two people	21
Doris - Moving up the bed, two people, pulling	22
Emma - Lateral transfer onto trolley	22
Moving out of a confined space on the floor	23
Care of Your <i>MaxiSlide</i>	24
Ordering Details	25

Foreward

Thank you for purchasing ARJO equipment.

Your *MaxiSlide*™ is one of a series of quality products designed especially for hospitals, nursing homes and other health care uses.

ARJO is dedicated to serving your needs and providing the best products available, together with training that will bring your staff the maximum benefit from every ARJO product.

Please contact us if you have any questions about the operation or maintenance of your ARJO equipment.

Conditions

- The unit is cared for in accordance with recommended, published “Operating and Product Care Instructions”.
- The product care, in accordance with ARJO requirements, must begin with the customer’s first use of the unit.
- The equipment is used for its intended purpose only and is operated within the published limitations.

- Only ARJO designated spare parts should be used.

Before using your *MaxiSlide*™, please read this manual thoroughly in its entirety. Information in the manual is crucial to the proper operation and maintenance of the equipment, and will help protect the product and ensure that the equipment performs to your satisfaction. Some of the information in this booklet is important for your safety and must be read and understood to help prevent injury. If anything in the manual is confusing or difficult to understand, please call ARJO Med. AB. Ltd. or its appointed distributor (the telephone number appears on the last page of this manual).

The following techniques are only a sample of the latest ergonomically correct patient handling manoeuvres carried out with the *MaxiSlide*™. There are many more shown on our CD-ROM *MaxiSlide*™ user guide which is available from your local ARJO office.

Safety Instructions

Symbols used adjacent to the text in these instructions:

Warning: Means: Failure to understand and obey this warning may result in injury to you or to others.

- **Note:** Means: this information is important for the correct use of this system or equipment.

See also “Care of Your *MaxiSlide*™” section.

The techniques shown are based on sound biomechanical principles. Pushing actions are carried out at a high level to avoid stooping and to allow use of the strong leg muscles. Pulling is carried out at a lower height, with the elbows straight, to avoid overuse of the shoulders and allow use of the leg muscles to achieve the movement. Where there is a choice, it is preferable to push rather than to pull.

Reference Mital:

- Pushing 80 -120cm off ground
- Pulling 60-80cm off ground (depending on your height)

Peak load is reduced by 50% if the manoeuvre is carried out slowly. This reduces the physical load experienced by the caregiver, and gentle movement is preferred by the patient.

Sliding sheets are generally removed once the manoeuvre is complete, but in some circumstances, e.g. at night, they may be left in place to allow patients more independent movement. In these circumstances, a suitable assessment must be carried out to ensure the patient’s safety.

Warning: Ensure that sliding sheets are never dropped or left on the floor, as they are extremely slippery and accidents could occur.

Full laundering instructions are included with the *MaxiSlide*™ but always ensure you follow your organization’s infection control policy regarding laundering, disinfection and reuse of this product.

- **Note:** Sliding techniques may be used successfully with patients of all dependency levels, but they must be subject to a suitable assessment. Factors you should consider are the environment (bed, space, type of sliding sheets), characteristics of the patient (ability to assist, size, medical condition, comprehension, level of any pain, weight), the task you are carrying out, the caregiver’s individual capability (never attempt to carry out a task which you feel exceeds your individual capability).

Using Your *MaxiSlide*

General

The *MaxiSlide*™ is a unique, ergonomically designed device having ultra-low friction properties, which will aid caregivers in moving or positioning patients while, at the same time, encouraging good posture without causing undue stress to either the patient or the caregiver.

The *MaxiSlide*™ will also encourage patients to help themselves wherever possible.

Warning: *MaxiSlide*™ has not been designed to, and therefore must not be used to lift patients from a floor or ceiling lift.

This document uses the following ARJO patient categories from the 'Resident's Gallery':

Carl: requires assistance with many of his daily activities.

Doris: requires a high level of assistance with all her daily activities.

Emma: is not capable of performing daily activities independently.

Albert and *Barbara* from the 'Resident's Gallery' can be found on the *MaxiSlide*™ CD Rom User Guide.

Carl, Doris & Emma - Fitting 2 MaxiSlide™s using the unravelling technique

Some patients may not be suitable for rolling, for example when they are in pain, or due to their size. In this instance, this technique may be used to insert two full length *MaxiSlide*™ sheets. The technique is first shown as a detail without the patient so it is easier to see what is happening.

Hold both *MaxiSlide*™ sheets with your outside hand to secure them and, with the inside hand palm up, unravel each fold, taking care to keep your hand in contact with the bed. Do not lift upwards. Unravelling the folds of the sheets away from the patient prevents skin shear.

Place both *MaxiSlide*™ sheets together lengthwise. Fold the two *MaxiSlide*™ sheets into 6-8" folds and then turn them over so that the folds are underneath.

Using Your *MaxiSlide*

It is recommended that you unravel the *MaxiSlide*™ sheets from the patient's head downwards, as this follows the direction of the patient's clothes and helps prevent the sheets from getting caught.

In case of pressure relieving mattresses, gently push down on the mattress to assist with fitting. You may also find it helpful to set the mattress pressure to a firm setting for the procedure.

Doris - Turning in bed by two people

This technique can be used with a patient who is unable to assist and, therefore, requires 2 caregivers to reposition in bed.

Ensure the bed is at the correct height for the fitting of the *MaxiSlide*™ sheets and, using the 'unravelling technique', place them underneath the patient, starting at the patient's head.

Using the principles of normal movement wherever possible, prepare the patient to turn to the side desired.

The patient is moved toward one side of the bed to allow space to turn onto her side. At this point, the bed is lowered, as the physical action to be undertaken is mainly pulling.

The caregiver grasps the *MaxiSlide*™ handles and adopts a 'walk' stance and, using her bodyweight, pulls the patient gently towards her.

Using the principles of normal movement, the caregiver then turns the patient towards the second caregiver using a gentle rolling movement. The patient may need to have her hips, shoulders and legs repositioned for comfort.

Remove both *MaxiSlide*™ sheets from behind the patient rather than the front, as this is where the majority of the slide material ends up following the technique.

Using Your *MaxiSlide*

Doris - Moving up the bed, two people, pulling

Use of a profiling bed can prevent a patient from slipping down toward the foot end. However, if such a bed is not available, the following technique can be used to slide the patient back toward the head end

This technique may be used when the bed is at a low height. Therefore, a pulling action is desirable. The patient is unable to assist with the movement.

The *MaxiSlide*™ sheets and tube have been placed beneath the patient to remove any 'shearing' effects of the movement.

Each caregiver stands on either side of the bed and holds the top *MaxiSlide*™ with the hand closest to the bed.

Ensuring good communication, the caregivers move simultaneously and slowly backwards to the head of the bed.

Warning: Care must be taken that the pulling action is not carried out too quickly as this may result in injury to the patient.

Emma - Lateral transfer onto trolley

Two *MaxiSlide*™ sheets are put in place using the 'unraveling' method. Use a Maxitube under the patient's feet if necessary.

- Note:** It is vital that you ensure that the trolley and the bed are of approximately equal height. It will however assist with the manoeuvre if the surface the patient is being transferred onto is slightly lower than the bed.

Ensure the bed and trolley castors are solidly locked.

Four Pull Straps are attached to the top *MaxiSlide*™, at shoulders, hip and feet. The Pull Straps prevent the caregivers from having to overreach.

The trolley is brought into place.

While the third caregiver rolls or tilts the patient towards him, the two caregivers place a 'pat slide' on the bed and slide it underneath the *MaxiSlide*™ sheets.

Using Your *MaxiSlide*

The two caregivers on the pulling side grasp the Pull Straps. The caregiver on the 'pushing' side will assist with initiating the transfer, taking care not to overstretch.

Using clear commands and working as a team, the two caregivers use a 'walk' stance and step back from the trolley, using their bodyweight to move the patient.

Final repositioning in the center of the trolley is achieved by using the handles on the *MaxiSlide*™.

Remove both slide sheets to complete the move.

Moving out of a confined space on the floor (any fallen patient)

On finding a patient on the floor, always ensure the correct health checks have been made prior to attempting to move him. If the patient is assessed as safe to move, this technique can be used to manoeuvre him into a space where a hoist can be used.

Place a pillow under the patient's head.

Using two *MaxiSlide*™ sheets and the 'unravelling' technique, place the sheets beneath the patient, ensuring they have been positioned beneath the pillow. This will protect the patient's head during the movement.

If space allows, attaching the Pull Straps will enable you to stand and work in a healthy posture.

If you require moving the patient over a greater distance, you can reposition the lower *MaxiSlide*™ in front of the direction of travel.

Care of Your *MaxiSlide*

Cleaning Instructions

The *MaxiSlide*[™], *MaxiTube*[™], *MaxiTransfer*[™] can be wiped clean using either a chlorine based solution (up to 10000 ppm), an alcohol based solution (up to 70%) or any good quality sterilization solution.

In addition, they can be washed at a maximum temperature of 80°C (175°F), and drip dried.

If necessary, a low temperature tumble dry can reduce drying times. However, the sliding sheets may develop a wrinkled appearance which should not affect performance.

Attention!

- *Please ensure that the MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] are always packed away or hung up after use in order to prevent people accidentally slipping on them.*
- *The MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] are not designed to be left directly under the patient. However, in the case where there is a need for long term patient positioning in bed, it may be possible, based on a clinical evaluation of the patient and using an interface such as a cotton sheet, to leave the product under the patient*
- *Please check the MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] regularly for any signs of wear and replace as necessary.*
- *The MaxiSlide[™], MaxiTube[™], MaxiTransfer[™] should only be used where no better alternative to moving or positioning the patient is available.*

Ordering Details

Ordering Details

Art. No.	Consisting of
NSA 1000	2 <i>MaxiSlide</i> TM , 1 <i>MaxiTube</i> TM , Operating Instructions all in a Resealable Bag
NSA 3000	2 <i>MaxiTransfer</i> TM , Operating Instructions all in a Resealable Bag
NSA 0510	8 <i>MaxiSlide</i> TM Straps
NSA 0500	1 <i>MaxiSlide</i> TM transfer sheet (purple)
NSA 6000	1 <i>MaxiSlide</i> TM XL transfer sheet (blue) 1 1/2 x std width
NSA 7000	1 <i>MaxiSlide</i> TM XXL transfer sheet (orange) 2 x std width
NSA 0600	1 <i>MaxiTube</i> TM

To Assist your staff further in other techniques of using your *MaxiSlide*TM, a CD-ROM or video is now available covering all the 'Resident's Gallery' from *Albert* through to *Emma*. For further details, please contact your local ARJO sales representative.

The CD-ROM and video cover the following uses:

- *Introduction (Resident's Gallery)*
- *Safe Use of the MaxiSlide*TM
- *Fitting & removing the MaxiSlide*TM & *MaxiTube*TM - *Bridging and Scissors Technique*
- *Independent moving up the bed - Knees bent, sitting position, using a trapeze, straight legs.*
- *Turning independently.*
- *Turning patient: the hip tilt.*
- *Moving sideways and sitting on the edge of the bed.*
- *Getting into bed - independently and assisted.*
- *Placing and removing full length MaxiSlide*TM
- *Unravelling Technique*
- *Moving up the bed with assistance*
- *Turning a patient away*
- *Turning with one caregiver from one side of the bed*
- *Turning in bed by two people*
- *Moving towards the head of the bed by two caregivers - pulling and pushing technique*
- *Turning into a prone position*
- *Lateral transfer onto a theater and bath trolley*
- *Placing a sling in a chair with a slide sheet*
- *Placing a stretcher sling between two MaxiSlide*TM's
- *Placing an X-Ray plate*
- *Moving out of a confined space on the floor (any fallen patient)*
- *Fitting a sling in a molded chair using a MaxiTube*TM
- *Using a MaxiSlide*TM to remove sling on a bed

*Placing a sling on a bed using MaxiSlide*TM's

AUSTRALIA

ArjoHuntleigh Pty Ltd
78, Forsyth Street
O'Connor
AU-6163 WESTERN AUSTRALIA
E-mail: sales@huntleighhealthcare.com.au
Tel: +61 89337 4111
Fax: +61 89337 9077

BELGIQUE / BELGIË

ArjoHuntleigh NV/SA
Evenbroekveld 16
BE-9420 ERPE MERE
E-mail: info@ArjoHuntleigh.be
Tel: +32 (0)53 60 73 80
Fax: +32 (0)53 60 73 81

CANADA

ArjoHuntleigh Canada Inc.
1575, South Gateway Road
Unit C
Mississauga
CA-ONTARIO L4W 5J1
E-mail: info@arjo.ca
Tel: +1 905 238 7880
Fax: +1 905 238 7881

CESKA REPUBLIKA

Arjo Hospital Equipment s.r.o.
Hlinky 118
CZ-603 00 BRNO
E-mail: arjo@arjo.cz
Phone: +420 549 254 252
Fax: +420 541 213 550

DANMARK

ArjoHuntleigh A/S
Vassingerodvej 52
DK-3540 LYNGE
E-mail: info@huntleigh-healthcare.dk
Tel: +45 4913 8486
Fax: +45 4913 8487

DEUTSCHLAND

ArjoHuntleigh GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
E-mail: info@arjo.de
Tel: +49 6134 1860
Fax: +49 6134 186 160

ESPAÑA

ArjoHuntleigh Ibérica S.L.
Carretera de Rubi, 88, 1ª planta - A1
Sant Cugat del Valles
ES-BARCELONA 08130
E-mail: promo@ArjoHuntleigh.com
Tel: +34 93 583 1120
Fax: +34 93 583 1122

FRANCE

Arjo Equipements Hospitaliers S.A.
45, Avenue de l'Europe
EUROCIT BP 133
FR-59436 RONCQ CEDEX
E-mail: info@arjo.fr
Tel: +33 (0)3 20 28 13 13
Fax: +33 (0)3 20 28 13 14

UK

Arjo Med AB Ltd.
St. Catherine's Street
UK-GLOUCESTER GL1 2SL
E-mail: uksales@arjo.co.uk
Tel: +44 1452 4282 00
Fax: +44 1452 4283 44

HONG KONG

ArjoHuntleigh (Hong Kong) Limited
Room 3A, 4/F Block B
Hoi Luen Industrial Centre
55, Hoi Yuen Road
Kwun Tong
HK-KOWLOON
Tel: +852 2508 9553
Fax: +852 2389 5797

ITALIA

ArjoHuntleigh S.p.A.
Via Tor Vergata 432
IT-ROMA 00133
E-mail: promo@ArjoHuntleigh.it
Tel: +39 06-87426214
Fax: +39 06 87426222

NEDERLAND

ArjoHuntleigh Nederland BV
Biezenwei 21, 4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0)344 64 08 00
Fax: +31 (0)344 64 08 85
E-mail: info@ArjoHuntleigh.nl

NORGE

Arjo Scandinavia AB
Ryenstubben 2
NO-0679 OSLO
E-mail: pot@arjo.no
Tel: +47 22 08 00 50
Fax: +47 22 57 06 52

ÖSTERREICH

ArjoHuntleigh GmbH
Dörrstrasse 85
AT-6020 INNSBRUCK
E-mail: office@arjo.at
Tel: +43 512 20 4160-0
Fax: +43 512 20 41 60 75

POLSKA

ArjoHuntleigh Polska Sp. z o.o.
ul. Ks. Wawrzyniaka 2
PL-62052 KOMORNIKI
E-mail, sales: international@arjo-huntleigh.com
Tel: +48 61 662 1550
Fax: +48 61 662 1590

SCHWEIZ / SVIZZERA

ArjoHuntleigh AG
Florenzstrasse 1D
CH-BASEL 4023
E-mail: arjo.sic@arjo.ch
Tel: +41 61 337 97 77
Fax: +41 61 311 97 42

SVERIGE

ArjoHuntleigh
Box 61, 241 21 Eslov
Sweden
Tel: +46 413-645 00
Fax: +46 413-645 83
Email: kundservice@arjohuntleigh.com

USA

ArjoHuntleigh
2349 W. Lake
St. Addison, IL. 60101
Tel: 1-800-323-1245
Fax: 1-888-389-ARJO(2756)
Email: info@arjousa.com

www.arjohuntleigh.com

© Medibo Medical Products NV
Heikant 5, 3930 Hamont Achel
Belgium

If your country is not listed here,
please contact your local distributor,
or ARJO International AB,
Box 61, S-241 21 Eslöv,
Sweden.

Tel +46 413 64500.
Fax +46 413 64563.

NSX01000-INT 1

ARJOHUNTLEIGH
GETINGE GROUP

GETINGE GROUP is a leading global provider of equipment and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences, equipment, services, and technologies are supplied under the brands ArjoHuntleigh for resident hygiene, resident handling and wound care, GETINGE for infection control and prevention within healthcare and life science and MAQUET for surgical workplaces, cardiopulmonary and critical care.