




HUMAN CARE

Your Life. Your Way.

Fitting Protocol

Sit Sling
Model 25030

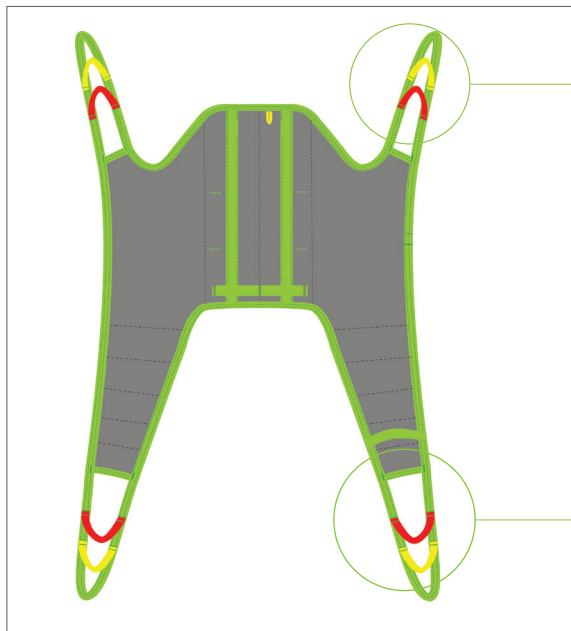

- ☐ Green Strap
- ☐ Yellow Strap
- ☐ Red Strap

Extension Loops

- ☐ 21180-10 (10cm/3.9")
- ☐ 21180-15 (15cm/5.9")
- ☐ 21180-20 (20cm/7.8")
- ☐ 21180-25 (25cm/9.8")
- ☐ 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbent
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle

- ☐ Green Strap
- ☐ Yellow Strap
- ☐ Red Strap

Extension Loops

- ☐ 21180-10 (10cm/3.9")
- ☐ 21180-15 (15cm/5.9")
- ☐ 21180-20 (20cm/7.8")
- ☐ 21180-25 (25cm/9.8")
- ☐ 21180-35 (35cm/13.7")

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 300kg/660lb

☐ **Model 25030 - Polyester**

Transfer Situation:




Available sizes:


☐ Indicate size required

JS **JM** **S** **M** **L** **XL**

Leg Placement:

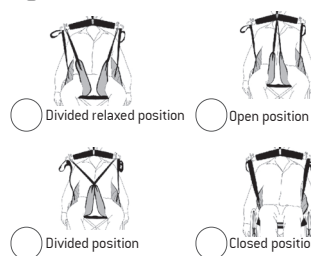

Suspension:

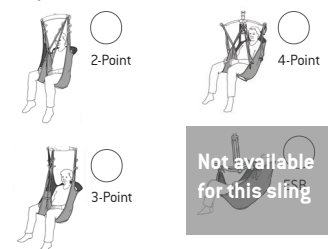

Accessories required:



- ☐ 21146 Leg Padding - Small
- ☐ 21146 Leg Padding - Medium
- ☐ 21146 Leg Padding - Large



- ☐ 25102 Reinforced Leg Support - One Size



- ☐ 50866 Upholstered Cover for Hanger Bar



- ☐ 25145.*
Waist Belt
[Velcro]

***Junior Sizes**
-40B (40cm/15.7")
-50B (50cm/19.7")
-60B (60cm/23.6")

***Adult Sizes**
-60 (60cm/23.6")
-70 (70cm/27.6")
-80 (80cm/31.5")
-90 (90cm/34.4")
-100 (100cm/39.4")
-110 (110cm/43.3")
-120 (120cm/47.3")
-130 (130cm/51.2")

Size (eg. -60B)

*only for Model #25146



- ☐ 25146.*
Waist Belt
[Safety Buckle]

Patient Information:

Name

Facility

Room

Lift Used

Hanger Bar Used

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Sit Sling is a shoulder high model with divided and reinforced leg parts as well as a reinforced back. It is designed to give the patient extra support and is a good solution for those users with poor trunk stability. Made of Polyester fabric, it is fast and simple to apply and can be used for most lifting situations. The Sit Sling creates an upright seating position which makes it ideal for proper positioning when transferring into a chair. The Sit Sling can be used with or without a back belt.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name	
Date	Position/Title



Important! This fitting protocol does not replace the user manual. Read all instructions before use!