



Pelvic Positioning Strap Installation on Raz AP, SP, & AT Rehab Shower Chairs (Z735 Option)

For more information please visit www.razdesigninc.com

Kit includes: two straps with “double D” rings and a plastic buckle

1. Take both straps and both “double D” rings out of the accessory box.
2. Weave the loose end of one strap through the “double D” ring and slide the “double D” ring several inches from the end.
3. Feed the strap through the slot in the backrest mounting plate – from the inside to the outside and then weave the loose end back through the “double D” ring.
4. Repeat steps 2 and 3 for the other side of the strap.
5. Adjust the “double D” ring and tighten the pelvic strap as needed.
6. Pull tightly on each strap to ensure that they are secured properly prior to use.

