


# The CHURCHiLL

**“Quick-Change”  
Incontinent Cover  
Instructions**



Merritt Manufacturing

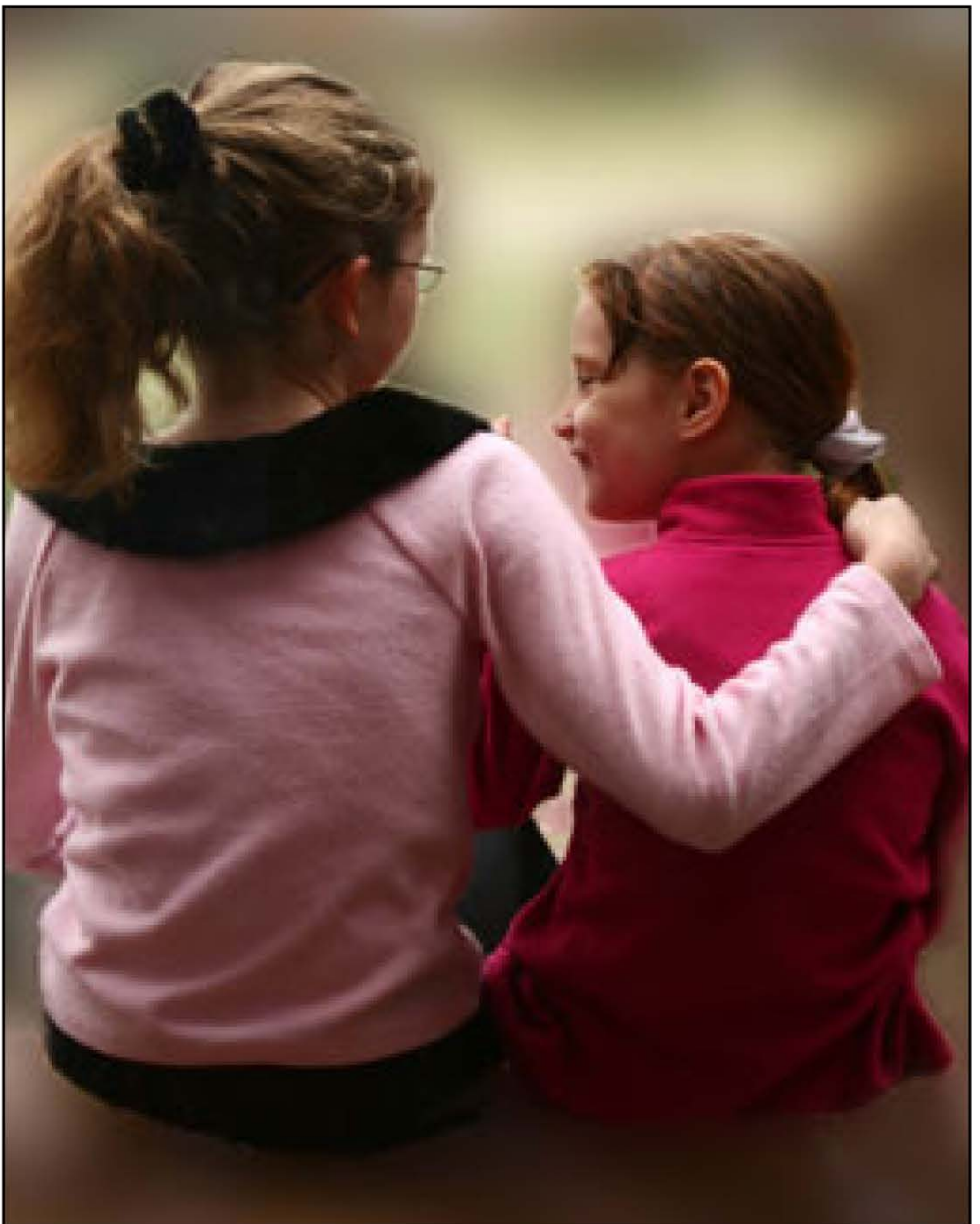

**WARNING!**  
**FAILURE TO COMPLY WITH THE  
FOLLOWING WARNINGS CAN RESULT  
IN DEATH OR SERIOUS INJURY TO  
YOUR CHILD:**

- The incontinent cover described in this instruction booklet is only to be used with the *Churchill* child safety seat manufactured by Merritt Manufacturing, Inc. **Do not use with any other child restraint system.**
- Prior to each use, verify that the installation of the incontinent cover is **not** creating any slack in the harness or buckle webbing.
- These are supplemental instructions which only describe how to use the incontinent cover. Please read and follow all instructions on this child restraint and in the written instructions located in the pocket underneath the restraint.



## Incontinent Cover Details

**Velcro Patches**



**Buckle  
Anchor Loop**



**Buckle  
Anchor Loop**



**Velcro Tabs**





# Installation of Incontinent Cover

- 1. Fit the incontinent cover around the front edge of the Churchill (Figure A).
- 2. Lift the front edge of the Churchill up until it is sitting vertically (Figure B).
- 3. Attach the velcro tab on the right side to its corresponding tab on the plastic base. (Figures B and C).
- 4. Attach the velcro tab on the left side to its corresponding tab inside the instruction pocket. (Figures D and E).



A





## Installation of Incontinent Cover, Continued

5. With the restraint upside down, grasp one of the belt guide anchors, twist it 90 degrees, and insert it (from the bottom side of the restraint) through the slot as shown in Figure F.



6. Pull the anchor completely through to the top side of the restraint (Figure G).



7. Find the buckle anchor loop (See Arrow-Figure H) on the same side of incontinent cover.



8. Insert the buckle anchor through the loop in the incontinent cover (Figure I).





# Installation of Incontinent Cover, Continued

9. Grasp the belt guide anchor, twist it 90 degrees, and insert it through the slot as shown in Figure J.



10. Pull the anchor completely through to the bottom side of the restraint (Figure K). See Figure L to see how it should look after the buckle guide is installed through the buckle anchor loop.



11. Attach the two attachment pads to the vertical straps as shown in Figures M and N.





12. Attach the incontinent cover by attaching the two attachment patches (on the incontinent cover) to the two attachment pads on the vertical straps as shown in Figures O, P, and Q.

