


Application: Use only with Reliant Stand Up Lift.

Models: RPS350-1, RPS350-2

STANDING SLING R130

The R130 and R131 standing slings are for residents who are partially dependent, have at least 60% weight bearing capacity, have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance), and are able to bend at the hip, knees and ankles. Use the R130 or R131 standing slings for standing assistance, quick toileting, weight bearing practice and for transferring resident from bed to chair, or chair to bed.

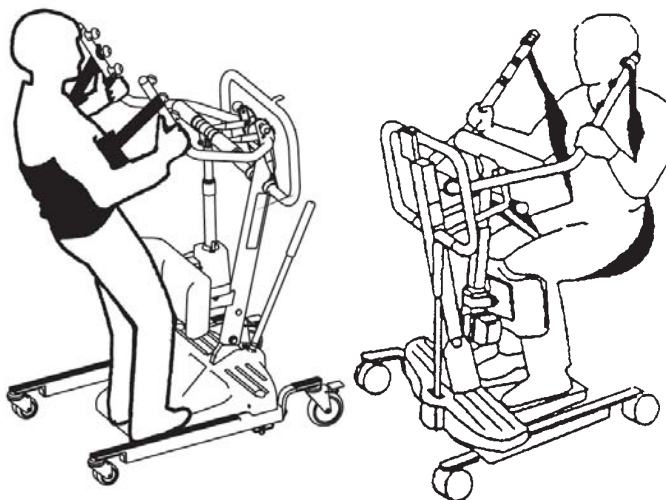

TRANSPORT SLING R134

The R134 and R136 transport slings are for transferring residents with minimal weight-bearing capacity, who have head and neck control, are cooperative, can sit up on the edge of the bed (with or without assistance), and are able to bend at the hips, knees and ankles. Use the R134 and R136 transport sling for transferring resident from bed to chair, chair to car or general in-room transport. Do not raise resident to a full standing position while using the transport sling.

Sling Binding Colors




Note: Sling size and fit can vary significantly depending on patient weight and girth. These are general guidelines. Consult physician before sling selection.

Washing instructions are on each sling label. Washing temperature should not exceed 180°F. Air-dry or dry at low temperatures.

All slings are constructed of polyester/-nylon fabrics which can be used in wet or dry environments.

Sling Sizes

Colour / Weight Chart

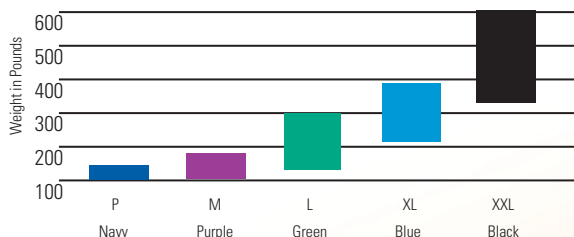

Invacare® Reliant Full-Body Slings with Commode Opening



Invacare® Reliant Toileting Sling with Belt



Invacare® Reliant Divided-Leg Slings with Headrest



Invacare® Reliant Transport Sling