



Immedia Transfer Boards

A WIDE RANGE OF SOLUTIONS FOR SEATED TRANSFERS

The transfer board is intended to bridge shorter gaps when performing seated transfers. The board's low friction upper surface allows the user to transfer independently or with assistance. Immedia Transfer Boards are used when transferring between bed and wheelchair / shower chair, between wheelchair / chair / toilet chair or to and from the car. All Transfer Boards have a non-slip surface on the underside. Combining a transfer board with a glide cushion provides even lower friction and is recommended when the transfer involves bare skin, for instance to and from a shower chair or toilet.



Immedia Butterfly

Immedia Butterfly is a long, wide and stable transfer board. It has cut outs for positioning against the wheel of a wheelchair and non-slip pads on the underside which allow for good positioning and avoid sliding out of place.

Immedia Butterfly GlideWing

Butterfly can be used with the special Butterfly GlideWing, which is mounted on the board. The GlideWing slides along the board and provides additional slide function.



Immedia 3B-Board's length allows transfer across wider gaps.

Immedia 3B-Board

Immedia 3B-Board is a stable transfer board with a traditional curved shape. The board can be placed either curve forward or curve backward depending on the transfer situation and surrounding obstacles, eg: the wheel of the wheelchair.

RECOMMENDED
USE!



1. Place the Immedia Butterfly close to the user's seat, and slide the GlideWing under one seat bone.



2. If possible, the user holds on to the opposite armrest when he/she starts to slide towards the wheelchair. Use the Immedia Pediturn to ease turning.



3. When the transfer is complete, remove the Immedia Butterfly and engage wheelchair armrests and legrests.

TIP!

When transferring with a board, ensure the surface you are moving to is slightly lower and use gravity to make the transfer easier.

Immedia E-Board

Immedia E-Board is a thin and flexible transfer board that allows easier application and removal.



Immedia E-Board with flap.

E-Board with flap

Immedia E-Board with flap has a flap along the side, which can be bent up or down. This provides more sturdiness and stability. The upward flap is used to protect against the wheel of the wheelchair. Available in two lengths.




Immedia E-Board without flap.

E-Board Compact

A smaller and lighter board that is easily carried and suitable for the more active and independent user. Immedia E-Board Compact is available in two lengths.



ONLINE
TUTORIAL!

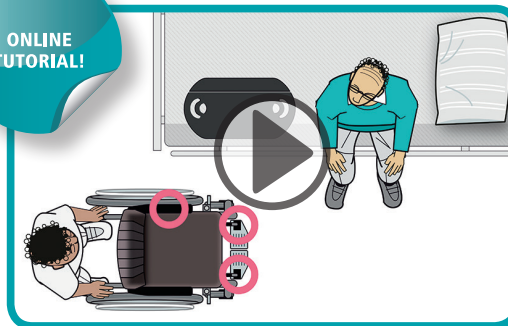

Learn more ...

... about how to use the transfer boards. Our new animated film *Transferboards – E-Board* gives you useful instructions for your daily work.



Immedia Dolphin

Dolphin is a sturdy and stable transfer board uniquely designed for transfer to the toilet. The board has cut-outs for positioning against the wheel of the wheelchair and non-slip pads, which allow good positioning and avoid sliding out of place. The cut-out allows the user to remain on the board while on the toilet.



Immedia Dolphin is uniquely designed for transfer to toilet.

INTENDED USE

- Transfer from bed to wheelchair / shower chair
- Transfer in and out of the car
- Transfer between wheelchair, chair, toilet, shower chair, commode

DESCRIPTION

WIDTH X LENGTH X HEIGHT

ITEM NO.

SPECIFICATIONS

IMMEDIA

- ~ 3B-Board, curved, white
- ~ E-Board Small, black
- ~ E-Board Medium, black
- ~ E-Board Wide, with flap black
- ~ E-Board Long, with flap, black
- ~ Butterfly
- ~ Butterfly GlideWing L
- ~ Butterfly GlideWing XL
- ~ Dolphin Toilet Board, white

CM

- 23 x 77
- 20 x 45 x 0,5
- 25 x 60 x 0,5
- 33 x 60 x 0,5
- 33 x 75 x 0,6
- 32 x 66
- 35 x 40
- 35 x 47,5
- Length: 69

INCH

- 9.1 x 30.3
- 7.9 x 17.7 x 0.2
- 9.8 x 23.6 x 0.2
- 13 x 23 x 0.2
- 13 x 29.5 x 0.2
- 12.6 x 26
- 13.8 x 15.7
- 13.8 x 18.7
- 27.2

ERP9557

ERP9558-01

ERP9558-02

ERP9558-03

ERP9558-04

ERP9559-00

ERP9559-03

ERP9559-04

ERP9572

Immedia E-board and 3B-board

MATERIAL: Polyethylene

Immedia Butterfly and Dolphin

MATERIAL: Fibreglass

Immedia Butterfly GlideWing

MATERIAL OUTSIDE & INSIDE: Nylon

PADDING: Polyester

WASHING INSTRUCTIONS

TRANSFER BOARDS: Clean with soapy water or disinfectant.



BUTTERFLY GLIDEWING: Wipe off with soapy water or disinfectant. Do not use fabric softener or bleach.

