
Glide sheets are used for safe and easy movement of patients in a bed or in a chair. The outer cover is Bucktex, a waterproof breathable material. The inner surface is slippery nylon.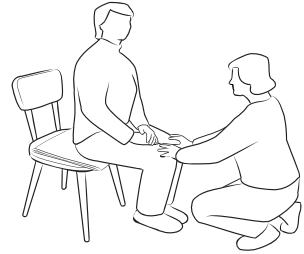

Uses: Turning patients while they are lying or sitting in bed.

Safe transfer from bed to chair.

Rotating patients from and into the prone position.

Sliding patient up in bed.

Benefit to the caregiver is also great, as easier moving of the patient reduces the strain on the caregiver's back.

Proper use of the glide sheet also prevents the effects of friction on the skin of the user.

Sizes available:

14A082 Glide sheet, 16" x 20" (41 x 51cm)

14A083 Glide sheet, 20" x 30" (51 x 76cm)

14A084 Glide sheet, 24" x 40" (61 x 102cm)


Precautions during use:

Make sure brakes are on (wheel)chair or bed before getting ready to transfer.

Make sure the sheet is inserted correctly and the colored sliding surface is on the inside.

Insert the sheet by 'log-rolling' the patient or asking the patient to lift themselves up from the surface. Whenever possible ask the patient to cooperate with the process.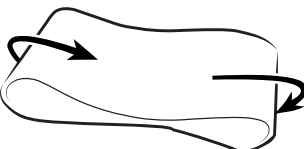


Remove the sheet by placing your hand between both layers of the glide sheet and invert by drawing the furthest bottom edge through the middle of the sheet. This effectively 'peels' the sheet out, without shearing effects.

Washing instructions:

Hand or machine-wash in 40 degrees Celsius. Hang to dry. Do not iron or expose to high temperatures. Do not bleach.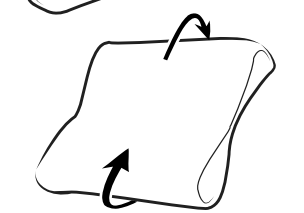

Note: Please use in accordance with your organization's policies and procedures. Do not leave the device near exposed or naked flames. Always ensure that this device is the appropriate equipment for the task of moving a particular patient.

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